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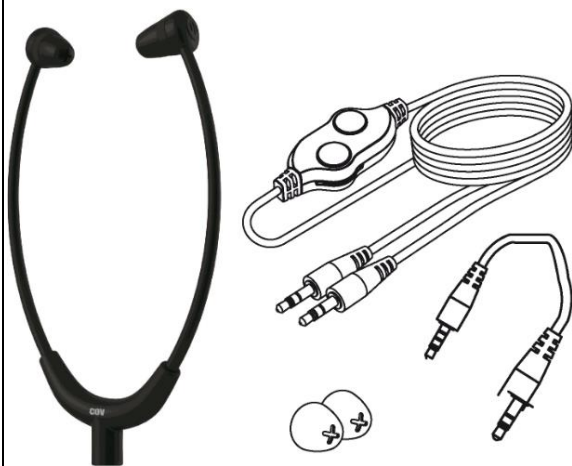
- Français

1. Accessoires inclus

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
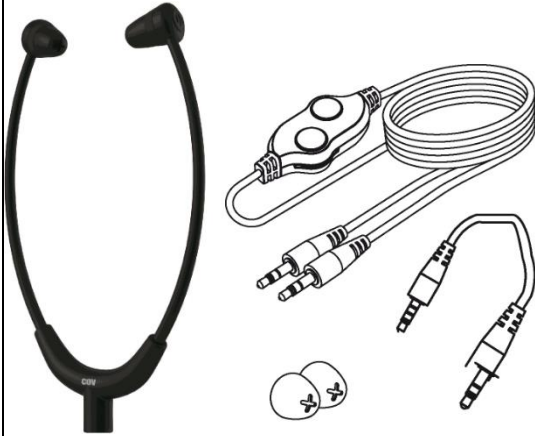
- English

1. Included accessories


1x stethoscopic headphones
1x 3.5mm jack / jack cable of 5m length
1x 3.5 mm jack / jack cable of 70 cm length
1x pair of earphone

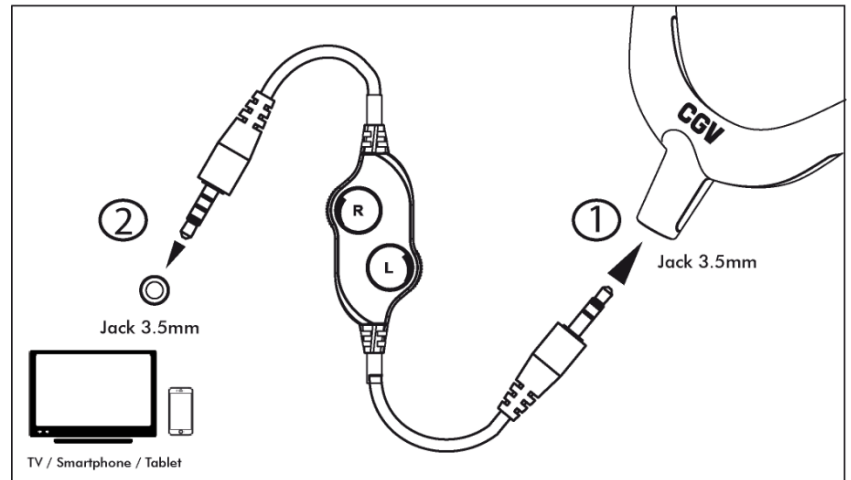2. How to install?

Just simply connect supplied audio cable between the headphone and a playing device to listen music or TV sound.

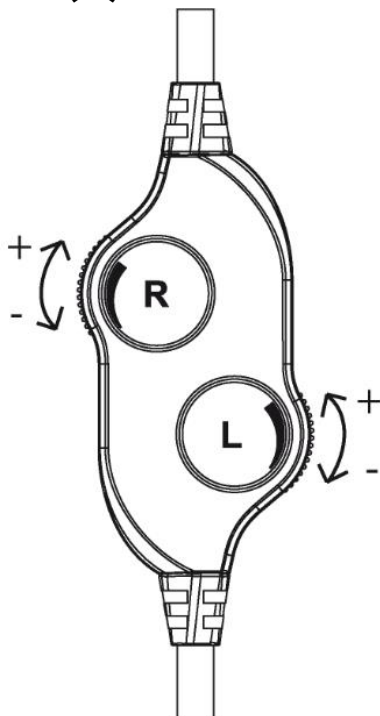3. How to adjust Left (L) and Right (R) Volume?

4. Technical specifications

Audio connection : 3,5mm (0,14 inch) audio jack
Speaker Drive Size: Dia. 9.20mm (0,36 inch)
Weight : 23g" (0,83 ounces)
Dimensions : 16,5 (L), 5,8 (l), 17,3 (H) cm
(6,49 x 1,96 x 6,69 inch)
Impedance: 16 Ω
Frequency: 20Hz ~ 20KHz

Safety instructions

- Please read this manual in its entirety before using your HEL DOLFIN PRIMO Headphones.
- Never open the device.
- Do not handle the unit with wet hands or in a very humid atmosphere.
- Never place a container of any liquid near on the headphones. Make sure that no liquid enters the unit. Never use liquid detergents to clean it.
- Never place heavy objects on the headphones even if they are not in use.

- The wireless audio headset is intended primarily for the hearing impaired, it can provide an extremely high level of sound. Prolonged listening of this headset to a sound level that is too loud may result in hearing loss.

- To prevent hearing loss, do not use headphones with high volume for a long time and adjust the volume to a safe level. The higher the volume, the shorter the listening time.

Waste Electrical and Electronic Equipment:

This symbol on the product, its accessories or documentation indicates that it complies with the European Directive **2012/19/EU**. This means that neither the product nor its used electronic accessories can be thrown away with other household waste. Please separate your used products and accessories from other waste. The civic disposal of your old product helps protect the environment and health, you will promote the recycling of the material that composes them in the context of sustainable development.

Individuals are invited to contact local authorities to find out about the procedures and collection points for these products for recycling.

The European RoHS Directive **2011/65/EU** of the restriction of the use of certain hazardous substances in electrical and electronic equipment also applies to this product.

MARKING FOR EUROPE:

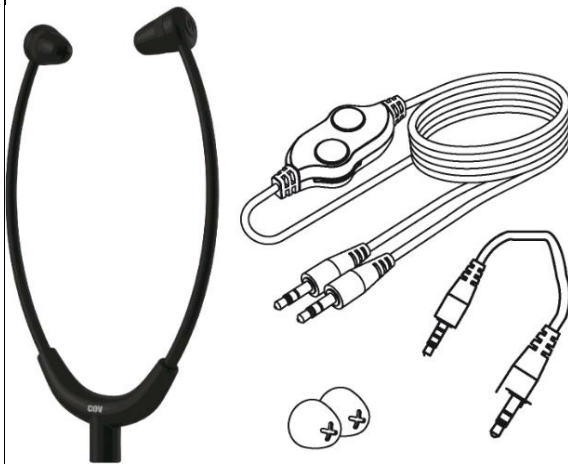
The CE marking attached to this product signifies its compliance with the Directives European **EMC (2014/30/EU)** and **LVD (2014/35/EU)**.

Note: Find the CE declaration of conformity here: <http://www.cgvforum.fr/>

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