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Dear Customer,

Thank you for purchasing this product. To be completely satisfied with your product, we recommend reading this user manual carefully.

If you have any difficulties or need more information, you can find our help centre by scanning the QR code. If you cannot find the answer to your question, you can also contact our After Sales Service: support.uk@terraillon.com

PRECAUTIONS

The following precautions should always be taken before using your TP251 blood pressure monitor, hereinafter referred to as “the device”.

GENERAL INFORMATION

1. Read the operating instructions carefully and keep them for future reference. If you pass this device on to others, please also pass on this manual.
2. Use this device only in accordance with the instructions in this user manual.

PRODUCT INFORMATION

- The blood pressure monitor is a digital monitor for measuring arterial pressure and heart rate with a wrist circumference of 13,5 cm to 19,5 cm. It is intended for adult use only.
- This device uses the oscillometric method. This non-invasive method is used to measure systolic pressure, diastolic pressure and heart rate. This technique detects variations in the brachial artery and interprets them digitally to facilitate reading.

- The wrist blood pressure monitor is suitable for use in the patient environment. That is, for indoor use at home.
- This device must be installed and started up in accordance with the information provided in this user manual.
- You can use this device to take your own measurements — no third party operator is required.
- Do not use this device for the diagnosis or treatment of a health condition or disease. Contact a healthcare professional in case of repeated abnormal measurements. Do not change your medication without the advice of your doctor or healthcare professional.
- Only use accessories approved by the manufacturer. The use of unapproved accessories may damage the device, injure users or provide incorrect data.
- Please use the device in the environment provided in the user manual. Otherwise, the performance and lifetime of the device will be affected and reduced.
- Similarly, performance can be affected by extreme temperature, humidity and altitude.
- Do not drop your device or expose it to strong shocks or vibrations.
- No modification of this equipment is permitted.
- Report any malfunctions or unexpected events to the manufacturer.
- Keep the device, charging cable out of reach of infants, children and elderly persons as they may present a choking or strangulation risk if used incorrectly.
- Keep small parts out of reach of children as they may swallow them. Contact a doctor immediately if ingested accidentally.

This device may only be used for the intended purpose described in this manual. The manufacturer cannot be held liable for incidental, consequential or special damage caused by misuse or abuse.

CONTRAINDICATIONS

1. The device uses the oscillometric method to measure arterial pressure. The wrist used for the measurement must have normal blood circulation. It is not suitable for people with restricted or impaired blood circulation.
2. Avoid taking measurements on people who are unable to communicate clearly, such as newborns, infants, children, or unconscious patients.
3. The device is not suitable for use on women who are or may be pregnant; people with diabetes; patients with implanted electronic devices such as cardiac stimulators, defibrillators, pacemakers, etc.; patients with pre-eclampsia, ventricular extrasystole, atrial fibrillation, peripheral arterial disease or kidney disease. Patients undergoing intravascular treatment or arteriovenous anastomosis; individuals who have undergone mastectomy.
4. The device is not suitable for measurements on people with illnesses that cause uncontrollable movements (shaking, chills, etc.).

If you are in any of these four situations or suffer from an illness, please consult your doctor before using the device.

- Similarly, if you are taking medication, consult your doctor to determine the appropriate time to measure your arterial pressure.
- Do not use this device if you are allergic to polyester, nylon or plastic.

PACKAGE CONTENTS

Wrist blood
pressure monitor
x1

Carry pouch
x1


Instruction
manual
x1


USB-C charging
cable
x1

PRODUCT OVERVIEW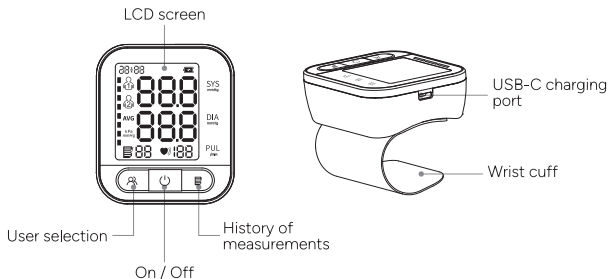

DATA READING

1. Date and time
2. User symbol
3. Average of the last 3 measurements
4. Arterial pressure unit
5. Colour indicator for arterial pressure (World Health Organisation classification)
6. Memory symbol
7. Saved measurements
8. Pulse symbol: heartbeat detection during measurement
9. Irregular heartbeat symbols: detection of irregular heartbeat
10. Heart rate (pulse): number of heart beats per minute
11. Diastolic pressure value: arterial pressure in the heart relaxation phase
12. Systolic pressure value: blood pressure in the heart contraction phase
13. Battery symbol

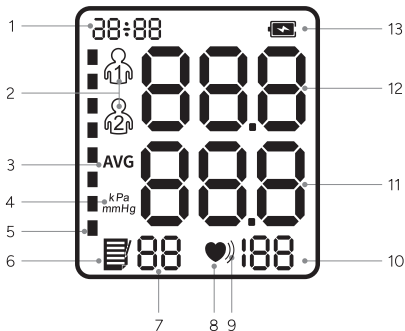Terraillon.

PREPARATION BEFORE USE

Before use, make sure that the device is operating safely and in good working order. Do not use if it is damaged in any way. The continuous use of a damaged device may cause injury, improper results or serious danger.

I. PRODUCT CHARGING

The TP251 wrist blood pressure monitor has a built-in rechargeable battery.

The battery must be charged before first use if the battery symbol  flashes.

1. Use the USB-C cable supplied with this device
2. Plug the USB-C plug into the USB-C charging port.
3. Connect the USB plug to a power adapter.
4. When the device is charged, the battery symbol  will stop flashing.

Charging time is approximately 2 hours.

Battery life is up to a maximum of 95 uses

Safety instructions for charging and protecting the device's lithium battery:

- For your safety, always use a 5V USB charger/adapter.
- Select an approved charger from a leading manufacturer. It must be certified and in good condition.
- DO NOT USE a quick-charge USB adapter, as this may damage the device.
- Only charge the device at a room temperature of between 10°C/50°F and 40°C/104°F.
- Do not expose the device to temperatures above 70°C/158°F. Do not throw into fire.

- Keep away from direct sunlight.
- Switch off the product after use.

NOTES:

We accept no liability for improper use of the product, e.g. if a higher voltage than specified is used.


Improper use can result in serious injury or damage to property.

In extreme cases, incorrect use of a lithium battery can lead to explosion, heat generation, fire or smoke.

II. USER SELECTION

This blood pressure monitor features two user profiles with a memory capacity of 99 records each.

When the device is switched off, press the button  corresponding to the selected user: user 1, user 2.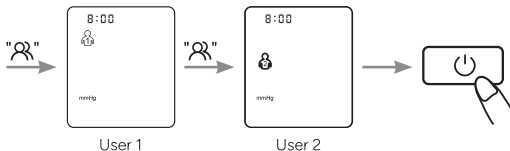


III. SETTING THE DATE AND TIME

Before taking your first measurement, set the current date and time on the wrist blood pressure monitor. (The date and time cannot be changed during a measurement.)

This information will be used when the data is saved in the memory.

1. With the device switched off, press the button  until the year flashes to enter set mode.
2. Press  to adjust the year, then press again  to confirm and move on to setting the month.
3. Press  to set the month. Then follow the same procedure to set the day / hour / minutes.
4. Press the button  one last time to confirm the settings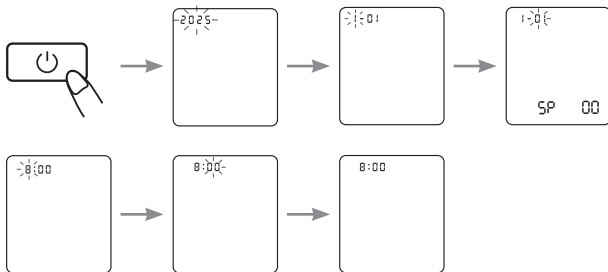


IV. UNIT DISPLAY SETTINGS

There are two units for arterial pressure: mmHg and kPa.

The default unit is mmHg.

When the device is switched off, press the button  for approx. 5 seconds to access the unit selection. Press the button  to toggle between mmHg and kPa, then press the button  to confirm the selection.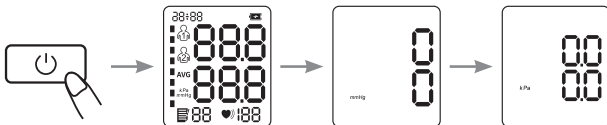


TAKING MEASUREMENTS

I. PLACING THE CUFF

The cuff is placed on the left wrist.

However, if your doctor has diagnosed you with poor blood circulation on this arm, or if you are in one of the situations explained in the “Warnings” section on page 35, you can use the right arm.

1. Remove all jewellery, such as watches and bracelets. Roll or push up your sleeve to expose the skin. Make sure that your sleeve does not squeeze your arm too tightly.
2. Place the cuff 10 or 15 mm from the bottom of the palm.
3. Secure the cuff with the Velcro fastener.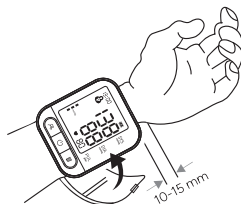

Terraillon.

Warnings

- Do not place the cuff on:
 1. Extremities other than the arm.
 2. A wound as this can lead to additional injuries.
 3. The arm of a person whose arteries or veins are undergoing medical treatment, i.e. intravascular access, intravascular therapy or an arteriovenous (A-V) shunt, which could disrupt blood circulation and cause injury.
 4. The arm on the same side as a mastectomy (especially when lymph nodes have been removed). It is recommended to measure on the unaffected side.
 5. An arm to which an intravenous drip or blood transfusion is attached.
 6. An arm where another monitoring device is applied. One or both devices may temporarily stop working if you try to use them at the same time.
 7. Patient areas where skin is delicate or damaged.
- Frequently check the cuff location for irritation or to ensure there is no irritation.

Safety:


In the rare event that the cuff remains fully inflated during the measurement, loosen and remove the cuff immediately. (High, prolonged pressure applied to the wrist may cause ecchymosis and discolouration of the skin).

While taking the measurement, check the air pressure on the LCD screen, it should not exceed 295 mmHg. If it does, press the button .

II. SITTING IN THE RIGHT POSITION

To take a measurement, you should be relaxed and comfortable in a room with a pleasant temperature.

- Sit upright in a comfortable chair with your back resting on the backrest.
- Do not cross your legs and keep your feet flat on the floor.
- Raise the arm to bring the cuff to your heart.
- Relax as much as possible.

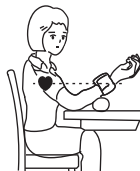III. MEASURING ARTERIAL PRESSURE

1. Press the button  to switch on the device and measure the arterial pressure
2. The blood pressure monitor will start to inflate for the measurement and display "00".

3. Inflating and measuring

The value shown on the display will increase, before slowly decreasing, and the heart rate symbol will flash.

4. Display and storage of results

Once the measurement is complete, the systolic pressure, diastolic pressure and pulse will be shown on the display.

Consult your doctor if you obtain unexpected results.

Terraillon.

Safety:

If you feel uncomfortable during the measurement, press the button  immediately to stop it.

If you experience any discomfort during a measurement, such as arm pain or other problems, press the button  immediately to release the air from the cuff.

Recommendations:

- Do not measure your arterial pressure immediately if you feel like urinating, after exercising, bathing or eating or after drinking tea, coffee or smoking a cigarette. Rest for 20 to 30 minutes before taking the measurement.
- Rest for 5 minutes before the first measurement.
- Stay still and do not speak during the measurement.
- Wait at least 3 minutes between each measurement.
- Measure your arterial pressure at approximately the same time every day.

IV. TURNING OFF THE DEVICE

Press the button  to switch off the blood pressure monitor. Otherwise, the monitor will switch off automatically after 20 seconds.

Warnings:

- Avoid using your blood pressure monitor simultaneously on the same limb. Similarly, too frequent arterial pressure measurement can disrupt blood circulation, cause injuries and display inaccurate measurements.

- Please ensure that the device does not cause any prolonged impairment of blood circulation during operation.
- The device contains sensitive electronic components. To avoid measurement errors, avoid taking arterial pressure measurements in the vicinity of an interference signal emitted by a strong electromagnetic field or a fast transient/burst electric signal.

Examples: high-frequency (HF) surgical equipment; magnetic resonance imaging (MRI); scanner, computer-aided tomography (CT), etc.

- Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations and walkie-talkies can cause interference that can affect the accuracy of measurements. A minimum distance of 1 foot (30 cm) from these devices must be maintained during a measurement.
- This device is portable, but not intended to be used during patient transport.
- This device is not suitable for continuous monitoring during emergencies or medical operations.
- This device is not AP/APG equipment. Do not use the device in the presence of flammable anaesthetics or in environments mixed with air or oxygen or nitrous oxide.

DATA MANAGEMENT

MEMORY FUNCTION

1. Each value measured is automatically stored under the previously selected user. This device can store up to 99 measurements for User 1 and User 2. When the memory is full, the old values are replaced by the new ones.

2. In off mode, press the button  once and the device will display the average value of the arterial pressure measurements of the last two or three readings. Press the button  again and the last measured value will be displayed.
3. Clearing the memory

When the device is switched off, press the button  and then  to select the user whose measured values must be deleted.

Press and hold the button  for 3 seconds to delete the stored data. The icon  will appear on the screen.

DETECTION OF AN IRREGULAR HEARTBEAT

The heartbeat icon  flashes as soon as a pulse is detected during measurement.

The icon  will be displayed if an irregular heartbeat is detected.

WHO ARTERIAL PRESSURE INDICATOR READING

For information, please refer to the pressure classification table (mmHg unit) established by the WHO and the International Society of Hypertension (ISH) in 1999.

Colour display		BP classification:	Systolic blood pressure (mmHg)	Diastolic blood pressure (mmHg)
	Red	Severe hypertension	≥ 180	≥ 110
	Red	Moderate hypertension	160-179	100-109
	Red	Mild hypertension	140-159	90-99
	Orange	High normal value	130-139	85-89
	Green	Normal arterial pressure	120-129	80-84
	Green	Optimal arterial pressure	< 120	< 80

Reminder: This device enables you to monitor your blood pressure.

It is not intended to be a diagnostic device. Always contact your doctor for advice, diagnosis and treatment.

SPECIAL INDICATIONS

ERROR SYMBOLS

If the measurement is abnormal, one of the following symbols may appear. Please use the recommended measurement method.

Error	Cause/Solution
Er U	Pressure cannot reach 30 mmHg (4 kPa) in 12 seconds.
Er H	Inflation reaches 295 mmHg and deflates automatically after 20 ms.
Er 1	Pulse is not detected correctly.
Er 2	Too many disturbances (movement, conversation or magnetic disturbance during a measurement).
Er 3	The measurement result is abnormal.
Er 23	The SYS value is below 45 mmHg.
Er 24	The SYS value is greater than 230 mmHg.
Er 25	The DIA value is below 25 mmHg.
Er 26	The DIA value is greater than 160 mmHg.

ANOMALIES & TROUBLESHOOTING

Fault	Inspection items
No power supply	Battery empty
Unable to measure due to display error	If the arm is moved during pressurisation If you talk during the measurement
Leakage at the cuff	The cuff is too loose
	If arterial pressure still cannot be measured after trying the solutions mentioned above, please contact the dealer. DO NOT disassemble the device yourself

SAFETY INFORMATION

	Caution
	TYPE BF APPLIED PART
	Symbol for marking of electrical and electronic equipment in accordance with the directive 2012/09 EU
	Refer to instruction manual
	Keep dry
	Low battery
	Keep out of direct sunlight
	Vertical upwards
IP21	2 protected against solid foreign bodies 12.5mm long $\varnothing$ and above; 1 protection against
CE 0123	CE

	Manufacturer
	Date of manufacture
SN	Serial number
LOT	Batch number
EU REP	Authorised representative in the European community
RoHS	RoHS Mark
MD	Medical appliance
UDI	Unique device identifier
	Importer in Switzerland
CH REP	Authorised representative in Switzerland
	Obligations to be complied with at all times
	Component cannot or must not be disassembled
	Prohibitions

MAINTENANCE & CARE

CLEANING

- Clean the device and cuff with a soft, dry cloth. If necessary, use a damp cloth and a natural detergent. Do not use alcohol, benzene or other harsh chemicals.
- When cleaning, make sure that you never immerse any part of the blood pressure monitor in order to prevent liquid from entering the device.
- Do not clean the cuff in a washing machine or dishwasher!
- Clean within 3 minutes each time. The number of consecutive cleans must not exceed three.
- The service life of the cuff may vary depending on washing frequency, skin condition and storage condition. Typical service life is 10,000 uses.

DISINFECTION

- The body of the device can be cleaned with a soft, clean cloth slightly dampened with 75% medical alcohol for disinfection.
- Avoid using disinfection methods such as high-temperature steam or ultraviolet radiation, as these can damage the device and reduce its service life.
- It is recommended that the display unit be disinfected before and after each use. The number of consecutive cleans must not exceed two.
- The blood pressure monitor must not be used again if liquid has penetrated and damaged the device or cuff.

STORAGE

- Make sure your device is clean and dry before storing it.
- Store the device and cuff in a clean, dry place and protect them from heat, lint, dust, direct sunlight and corrosive gases. Never place heavy objects on top of it. Avoid places where there is a high risk of vibration, shock, moisture or exposure to splashes, as this may damage the device.
- Remove the cuff from your device and fold it gently.

Terraillon.

- Make sure that the rubber hose of the cuff is not crushed, stretched or kinked during storage.
- Store your device in its original cover or packaging. Store indoors at temperatures between -20°C and 55°C, ≤93% relative humidity, and 90 kPa to 105 kPa atmospheric air pressure. Make sure that it is stored in a well-ventilated area, free of corrosive gases.
- It may take up to 30 minutes for the device to warm up/cool down from the minimum/maximum storage temperature before it is ready for use.

MAINTENANCE

- No calibration is required.
- Do not attempt to repair the device yourself if it is not working properly. Repairs must only be carried out by authorised service centres.
- Do not use the device during maintenance or service.

WARRANTY AND ENVIRONMENTAL PROTECTION

This appliance is guaranteed against material and manufacturing defects for two years. During this period, the product will be replaced free of charge (proof of purchase must be presented if the product is under warranty). This warranty does not cover damage resulting from accidents, improper use or negligence. If you have a complaint, please contact the store where you purchased your product first.

Waste electrical products should not be disposed of with household waste.

Please recycle where facilities exist. Check with your local authority or retailer for recycling advice.

TECHNICAL CHARACTERISTICS

Model	WRS-35G	
Display	LCD	
Measurement method	Oscillometric measurement	
Measurement part	Wrist	
Measurement range	Arterial pressure	0~295 mmHg (0~39.3 kPa)
	Pulse frequency	40~199 bpm
Maximum pressure protection	295 mmHg (39,3 kPa)	
Accuracy	Arterial pressure	± 3 mmHg ($\pm 0,4$ kPa)
	Pulse frequency	$\pm 5\%$
Memory	2 users x 99 memories	
Power source	3,7V rechargeable battery	
Dimensions	77,6 mm (L) x 67 mm (l) x 28,8 mm (H)	
Screen size	40,6 mm (L) x 47 mm (l)	
Cuff size	Approx. 135 mm - 195 mm	
Weight	126 g	
Automatic shut-off	20 seconds	
Degree of protection against electric shock	Type BF	
Protection against ingress of water or particles	IP22	
Protection against electric shock	Internal power supply	
Blood pressure monitor service life	5 years	
Operating mode	Continuous run mode	

Operating environment	Temperature: 5°C ~ 40°C Humidity: 15% ~ 90% RH Air pressure: 80 kPa ~ 106 kPa
Transport and storage conditions	Store in a clean and safe place. Storage temperature: -20°C ~ 55°C Humidity: ≤93% RH Air pressure: 90 kPa ~ 105 kPa Store away from corrosive gases and in a place with good ventilation.

FAQS

What are systolic pressure and diastolic pressure?

Arterial pressure is produced by the heart beating as it contracts and relaxes. The maximum (systolic) and minimum (diastolic) arterial pressure can be measured.

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- The difference in arterial pressure between home and hospital measurements is approximately 20 mmHg - 30 mmHg (2.7 kPa - 4.0 kPa). This is because people tend to be more relaxed at home than in the hospital.
- Also, when the device is placed above the heart, the arterial pressure value tends to be much lower than it actually is. Make sure that the device is positioned at the same level as the heart.
- Pressure varies throughout the day, depending on time, physical effort and emotions. Furthermore, «white coat syndrome» often causes the pressure to rise when taken by a healthcare professional.

- The antihypertensive drug may have lost its efficacy. Please follow your doctor's instructions.
- The cuff may not be in the correct position. If the cuff is not placed correctly, no arterial pressure value will be obtained, and the arterial pressure value may be much higher than it is. Therefore, position the cuff correctly.
- The cuff is not tight enough. If the cuff is too loose, the compressive force may not be transferred to the artery and the arterial pressure value may be much higher than it is. Therefore, readjust and tighten the cuff further.
- The patient is not sitting correctly during the measurement. It is not advisable to squat, tilt, lean over or sit with legs crossed when taking arterial pressure due to increased abdominal pressure or the position of the wrist below the heart. Please take the measurements in the correct position.

Why does my arterial pressure change during the day?

1. Arterial pressure is not stable, it varies during the day. It is therefore important that the measurements are taken under identical conditions: regularly and at a fixed time, in quiet conditions and in the same environment.
2. If you are taking medication, the measurements may vary further.

When can I get better measurements?

- It is best to take the measurements in the morning, just after urinating, or when your mind and body are stable. We recommend taking the measurements at the same time of day each time.

Why is the arterial pressure value measured different each time?

- During systole, the arterial pressure varies to a certain extent. For example, a person with a pulse rate of 70 beats per minute will see their arterial pressure vary from 100 to 800 per day. Since the arterial pressure changes constantly, it is difficult to obtain the correct arterial pressure value by measuring it only once. Therefore, the measurement should be taken two to three times. The first measurement will usually be higher due to nervousness or inadequate preparation; in the second measurement the nervous

emotion will be slightly attenuated, so that the second measurement will typically be 5 mmHg to 10 mmHg (0.7 kPa to 1.3 kPa) lower the first time. This phenomenon is more obvious for people with high arterial pressure.

- When measuring continuously, please note that: There may be extravasation of blood because the arm is squeezed, causing blood from the fingertip to not flow smoothly.
- If you continue the measurement in the event of blood extravasation, you will not be able to obtain the correct value measured. Loosen the cuff, lift your hand over your head, grasp and stretch the left and right palms 15 times in a row. The extravasated blood can then be removed and you can continue measuring your arterial pressure.

Are the results the same with the right arm?

- Arterial pressure may be measured on both arms, but the results may differ from person to person. We therefore recommend that you always measure your arterial pressure on the same arm.

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EMC GUIDANCE

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Table 1

Guidance and manufacturer's declaration – electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not applicable
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable

Table 2

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity Test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4kV, ±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency 50Hz/60Hz magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz – 2,7 GHz 80 % AM at 1 kHz
NOTE Ur is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer's declaration - electromagnetic immunity							
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Modulation (W)	Distance (m)	IMMUNITY TEST LEVEL (V/m)
	385	380-390	TETRA 400	Pulse modulation 18Hz	1.8	0.3	27
	450	430-470	GMRS 460, FRS 460	FM $\pm$ 5kHz deviation 2kHz sine	2	0.3	28
	710	704-787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9
	745						
	780						
	810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0.3	28
	870						
	930						
	1720	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4,25; UMTS	Pulse modulation 217Hz	2	0.3	28
	1845						
	1970						
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	2	0.3	28
	5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	0.2	0.3	9
	5500						
	5785						

Terraillon.

Terraillon.

Terraillon SAS France & Headquarters

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Terraillon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay – Hong Kong
Tel: +852 (0)2960 7200
customerservice@terraillon-asia.com

Terraillon Corp USA

contact@terraillon.fr

Terraillon®

www.terraillon.com