BÉABA®

TOMYDOO

- FR Notice d'utilisation
- EN Instructions
- NL Handleiding
- DE Gebrauchsanweisung
- ES Folleto de Instrucciones
- PL Instrukcja obsługi
- RU Инструкция по эксплуатации
- عربي - دليل الاستخدام العربية

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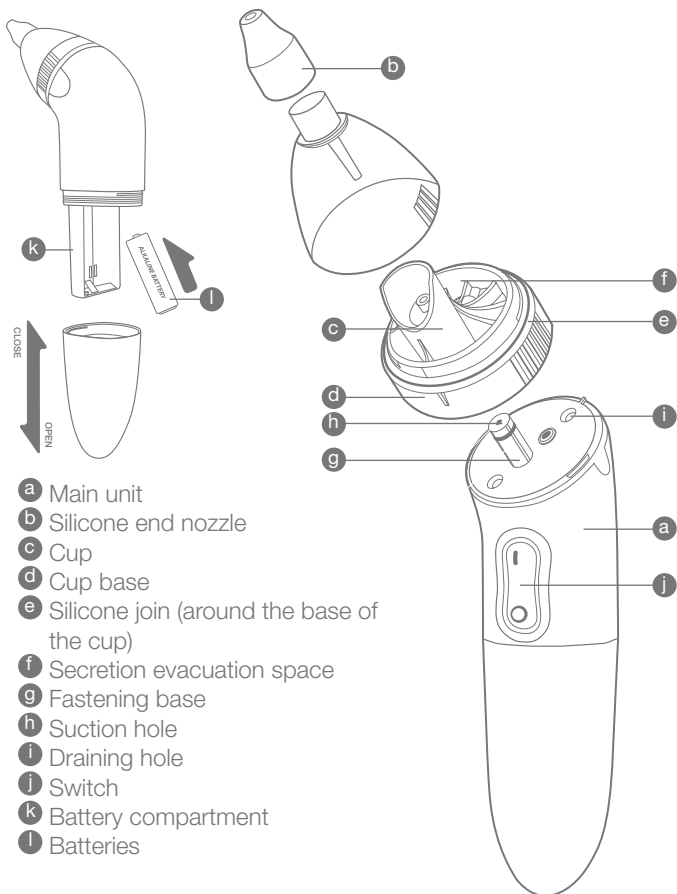
PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
AND KEEP FOR FUTURE REFERENCE.

Supplied with 3 tips:

- transparent for heavy secretions
- blue for normal secretions
- white for babies from 0 to 3 months

EN

Congratulations and thank you for your purchase: you now own a device which offers you unique performance.

RECOMMENDATIONS

- To improve the efficiency of the appliance, it is recommended to block one nostril while using it on the other.
- A few seconds are enough to remove secretions. Do not use the nose cleaner in the case of nasal irritation.
- It is recommended to clean the nozzle part between each use.
- Store the product in a dry place at room temperature. If not used for several months, it is recommended to remove the batteries.
- The Tomydoo nose cleaner helps prevent respiratory complications. In the case of infection, please consult a doctor.

N.B.: please keep the case out of reach of children and babies.

INSERTING THE BATTERIES

open the battery compartment (k) by removing it from the nose cleaner (a). Insert two LR3 batteries then shut the compartment.

SWITCHING ON

press the upper part of the switch (j) and the lower part to stop the vacuum.

USING THE APPLIANCE

position the silicone nozzle (b) just at the opening of the obstructed nostril for a few seconds. The secretions are collected in the secretion space (f).

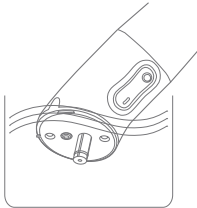
CLEANING AFTER USE

Separate from the main unit

- Pull the silicone nozzle (b).
- Pull the upper part including the cup base (d) to remove it from the rest of the main unit of the nose cleaner (a).
- Unscrew the cup (c) for cleaning.
- Make sure that the silicone joint is in the right position (e).
- Screw back and reassemble the different parts.

Nozzle part

Remove the nozzle part from the nose cleaner (b, c, d, e, f) and rinse the different parts with water. They can be sterilised with boiling water or steam.

Main unit

Make sure that the battery compartment is shut. After having removed the nozzle part, immerse the suction hole (h) of the appliance in water and switch on. In a few seconds the appliance is clean.

WARNINGS

Contains small parts. For security reasons, keep out of the reach of children.

Do not use the supple newborn tip beyond 3 months of age, to prevent any risk of it becoming stuck.

The logo which appears on this product of a crossed out wheellie bin indicates that the product should not be disposed of in landfill with normal household waste.

In order to avoid damaging the environment or human life, please separate this product from other waste in order to ensure that it is recycled in an environmentally safe manner.

For further details on existing collection sites, please contact the local administrative authorities or the retailer from which you purchased this product.

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