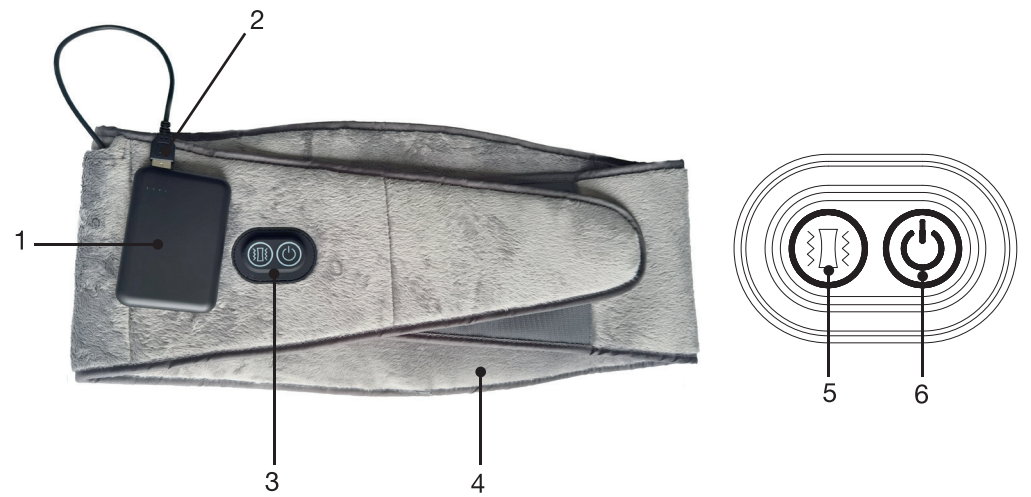
essentiel 

**Ceinture lombaire chauffante sans fil /  
Cordless heating lumbar belt /  
Cinturón lumbar inalámbrico de calor /  
Kabelloser Rückengurt mit Wärmefunktion /  
Draadloze verwarmende buik-/rugband**

**ECL3**

**Notice d'utilisation / User guide / Instrucciones de uso /  
Bedienungsanleitung / Gebruiksaanwijzing**

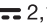
	<b>FR</b>	<b>EN</b>	<b>NL</b>	<b>ES</b>	<b>DE</b>
1	Batterie externe	External battery	Externe batterij	Batería externa	Externer Akku
2	Connecteur d'alimentation	Power supply connector	Voedingsconnector	Conector de alimentación	Netzschalter
3	Touche de commandes	Control buttons	Bedieningstoetsen	Teclas de control	Bedientasten
4	Ceinture chauffante	Heating belt	Warmtegordel	Faja térmica	Wärmegürtel
5	Vibrations	Vibration	Trillingen	Vibraciones	Vibrationen
6	Marche/arrêt + Chauffage	On/off + Heating	Aan/uit + Verwarming	ON/OFF + Calentamiento	Ein/Aus + Heizen

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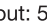
## your product

### Box contents

- 1 heating belt
- 1 external battery
- 1 instruction leaflet
- 1 usb-C charging cable

### Technical characteristics

- The belt will automatically shut-off after 1 hour
- The vibrations will automatically stop after 15 minutes
- Velcro fastening system
- Built-in control buttons: on/off - vibration
- 3 modes:
  - heating mode + light vibration
  - heating mode + strong vibration
  - heating mode only
- This belt should only be used with the battery HYY-02 provided
- Low battery indicator
- Battery level indicator
- Average charging time: 3 hours
- Over 2 hours of battery life
- Model: 8009621
- Input: 5 V  2.1 A
- USB output: 5 V  2.1 A
- Battery capacity: 5,000 mAh / 3.7 V
- Battery output voltage: 5 V / 2.1 A
- Product dimensions: 1365 x 222 x 20 mm
- Weight: 370 g
- Non-detachable cover/case

## use

- Press the  button for few seconds to switch on the device; it is automatically set to heating mode.
- Press the  button; the device will go into heating + vibration mode. Press the button again to select another vibration function. To stop the vibration, press the button once or twice depending on the vibration speed.
- The heating belt will automatically shut-off **after 1 hour**. To switch off the vibration mode, press the button ; otherwise, it will automatically stop **after 15 minutes**.

## maintenance and storage

**WARNING!** Before any cleaning or maintenance is performed, ensure the device has been unplugged from the battery and left to cool down.

### Cover maintenance

- Never use detergents or abrasive products to clean the device.

• Do not tumble dry

• Do not iron

• Do not dry clean.

• Do not bleach.

• Hand wash

- In the event that it is strictly necessary to clean the heating belt, unplug the external battery.
- Hand wash the surface of the heating cover, taking care not to submerge the device in water.
- It is recommended that you do not wash the cover too frequently. It should not be washed more than five times during its life to avoid completely damaging the internal components.
- To avoid damaging the belt, dry it flat on a clothes line without using clothes pegs or similar items to attach it.

## Belt maintenance

- Store the product in a cool, dry place.
- Do not hang the device by its power cord or wires.
- Do not directly expose the product to sunlight or high temperatures.
- Always remove the battery before storing the product.

WARNING: Allow the product to cool before storing it. To avoid making any noticeable creases in the product, do not place any objects on it when it is stored.

## troubleshooting

If you encounter the following problems, please refer to the solutions below. If the product still does not work properly or no longer works at all, please return it for a refund or exchange.

Problems	Possible causes	Solutions
Neither the on button indicator lamp nor the battery indicator lamp light up when: <ul style="list-style-type: none"><li>• the battery is connected to the product and</li><li>• you press the power button.</li></ul>	<ul style="list-style-type: none"><li>• The battery is low.</li><li>• The safety switch has been triggered due to overheating.</li></ul>	<ul style="list-style-type: none"><li>• Charge the battery.</li><li>• Allow the product to cool to room temperature and turn it back on.</li></ul>
Neither the on button indicator lamp nor the battery indicator lamp light up when: <ul style="list-style-type: none"><li>• the fully charged battery is connected to the heating belt and</li><li>• you press the power button.</li></ul>	The product or battery is defective.	Contact customer service.

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All information, designs, drawings and pictures in this document are the property of SOURCING & CREATION. SOURCING & CREATION reserves all rights to its brands, designs and information. Any copy and reproduction through any means shall be deemed and considered as counterfeiting.

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#### Protection of the environment

This symbol attached to the product means that it is an appliance whose disposal is subject to the directive on waste from electrical and electronic equipment (WEEE). This appliance may not in any way be treated as household waste and must be subject to a specific type of removal for this type of waste. Recycling and recovery systems are available in your area (waste removal) and by distributors. By taking your appliance at its end of life to a recycling facility, you will contribute to environmental conservation and prevent any harm to your health.

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#### \* Tested in our laboratories

Warranty valid from the date of purchase (receipt as proof of purchase). This warranty does not cover defects or damage caused by improper set up, incorrect use, or normal wear and tear of this product.

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