WITHINGS

PRODUCT GUIDE

EN
FR
DE
SV
NL
FI
DA
IT
ES
CS
PL
PT
RO
HU
SK
ET
EL
IS
LV
LT
SL
TR
BG
HR
NO

Box content

Withings Thermo

2 AAA 1.5V
(LR03) batteries
(already inserted)

Protective cap

Quick Start Guide
and product guide

Important Notice

Before using Withings Thermo, review the information in this guide. You may also find this guide online at: <https://www.withings.com/guides>

Retain this documentation for future reference. Installation instructions are available within the Quick Start Guide provided with this user guide. Please contact Withings when in need of assistance for setting up, using the device or to report unexpected operations / events. Any serious incident that has occurred in relation to the device should be reported to Withings and the competent authorities in your country of residence.

Manufacturer:
YA HORNG ELECTRONIC CO., LTD.
No. 35, Shalun, Anding Dist., Tainan City, Taiwan

Note to Canadian users

Health Canada does not recommend using non-contact infrared thermometers on children under 2 years of age. For older children, Health Canada recommends that infrared thermometers should not be used to manage important health concerns.

Intended use

The device is intended to measure temperature in the adult, children and baby populations. This device is not suitable for children born before term. Withings Thermo (SCT01) is intended for the intermittent monitoring of human body temperature at home.

Supported devices & Minimum requirements

In order to use your Thermometer, you need an iOS (14.0 or higher) or Android (8.0 and higher) device to install it. Thereafter, the product can be used without your mobile device on you, thanks to the Wi-Fi and Bluetooth®. For Bluetooth connection, you will need your phone to synchronize your results to see them in the Withings Application.

Precautions before use

- The device and the patient should stay in the same ambient temperature room for 10 minutes prior to taking a measurement.
- Infants' body temperature might vary more than adults'. Avoid taking measurements on babies after nursing or while they are crying. It is recommended to take measurements on children when they are calm.
- Do not take measurements over irritated skin or scars.
- For children under 3 months, perform 3 measurements in a row. If the 3 measurements are different, always take the highest one.
- If the patient has taken a bath or exercised, please wait for 15 minutes before taking a measurement.
- If the thermometer was stored in a cool or a warm room, wait 15 minutes before taking a measurement.
- When taking multiple measurements, wait one minute between each one
- Move hair out of the way and dry any sweat when taking a measurement

Important safety information

Self-diagnosis of measurement results and self-treatment are dangerous. Pregnant women should consult a doctor before using the thermometer. Please consult your doctor when there is a temperature increase on:


- Neonates and babies under 3 months
- Patients over 60 year old
- Immunocompromised patients
- Bedridden patients
- Transplant patients

Please consult your doctor if other symptoms (vomiting, diarrhea, pain, shivering, stiff neck....) occur even if there is no fever. This device is a precision thermometer measuring equipment liable to be understood by lay users but it still should be handled with care.


Warnings & safety notice

- Self-diagnosis and self-treatment can be dangerous. Do not interpret or take clinical action solely based on the assessment result without consultation of a qualified healthcare professional.
- Do not use the device on premature babies (babies born before term).
- Keep the device out of the reach of children and pets.
- Do not leave the thermometer unattended with infants or persons who cannot express their consent.
- This device is for personal use at home only.
- The sensor lens is fragile: do not touch it with your fingers
- Do not expose the device to humidity or water. In case the unit is wet, wait until it is dry.
- To clean the lens, use a soft dry cloth to wipe it or a cotton bud. For cleaning instructions, please see the cleaning section of this manual.
- Never try to disassemble or repair the product.
- The device does not provide alarms
- Misuse of the device can cause injury. Do not use the thermometer if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not try to repair or modify this equipment yourself. No service or repairs should be performed on the device other than the maintenance listed in the device's instructions for use.
- Do not use the thermometer for any purpose other than measuring temperature.
- Do not operate when the infrared optical lens is found to be soiled or damaged.
- Do not shake the unit violently.
- Do not use the device after a strong shock, such as dropping the unit on the floor.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Do not interconnect this equipment with other equipment not described in the instructions for use. Follow operational and storage conditions indicated in this user manual. Otherwise, measurement accuracy may be impacted. The user is an intended operator. Measurements can be affected by extreme temperatures, humidity, & altitude. Do not use accessories, detachable parts, or materials not described in the product guide. Use of parts and components other than those specified or provided by Withings could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Exposing the device to prolonged lint, dust or sunlight might reduce its life or damage it. A damaged sensor may lead to incorrect measurements.

How to take a measurement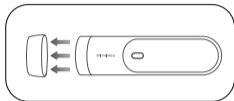

1. Remove the green protective cap.

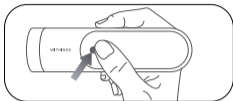
2. Press the button to turn the thermometer on.


3. Start from the middle of the forehead, slowly scan straight across to the hairline, up to 1cm away from the skin.

4. Thermo will vibrate twice when the measurement is complete. The temperature is displayed on the device. The unit is either in °F or °C. You can change this unit in the Withings app settings.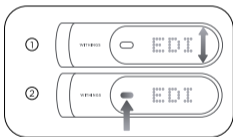

5. To assign the temperature to a user, scroll through names on the touch-sensitive zone (1). Press the button to assign the reading (2).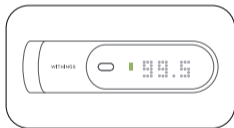


6. A colored LED indicates the fever level according to the user's age.

LED Colors meaning:

- No fever
- Mild fever
- High fever

Understanding the results

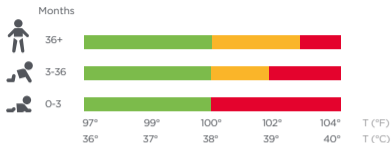Troubleshooting

If one of the following problems occurs while you are using Withings Thermo, refer to this section to help solve the problem. If the problem persists, please contact our customer service. For any enquiry, the serial number must be provided. It can be found inside the battery compartment. It is made of 12 characters in groups of 2, each group separated by a colon, for instance XX:YY:XX:YY:XX.

Problem

Solution(s)

Blank display. The device does not turn on.

1. Remove the AAA alkaline batteries and wait one minute.
2. Then reinstall the AAA alkaline cells back ensuring they are in the correct polarity.
3. Replace the AAA alkaline cells with new ones.

“Low battery icon”

Replace the AAA alkaline cells.

“RETRY”

The temperature is outside the range. i.e lower than 35°C (95°F) or higher than 43.2°C (109.76°F). Take a new temperature measurement, referring to the user manual. If the issue persists, contact customer support.

The temperature too rather high

The unit has been stored in a cold room. Leave the device in the room where the patient is for 30 minutes prior to the next measurement

The temperature seems too low	The skin has sweat on it or there is hair over the skin. Ensure that there is no hair in front of the sensor. Clean the skin with a dry cloth and wait for 5 minutes prior to taking a measurement. The patient has been in a cold room. Wait until the patient is warmer before taking a measurement. Measurement was not taken on the temple. Please refer to the user manual to place the sensor on the correct location.
Bluetooth does not seem to work	Smartphone is out of range. Please bring your smartphone closer to your device. Smartphone's Bluetooth is OFF. Please switch your smartphone's Bluetooth ON.
Wi-Fi does not seem to work	Device is out of range from the Wi-Fi source. Please get your smartphone and device closer to your Wi-Fi source.

After use

Cleaning:

Do not use an alcohol-based agent or solvent to clean the device.

Clean the device with a soft and dry cloth.

Do not submerge the device or any of the components in water.

Ensure that no liquid enters the interior of the thermometer.

Do not dismantle the device or try to repair it yourself.

After cleaning, wait for 10 minutes before taking a temperature measurement.

Storage:

- Store the device and the components in a clean and safe location.

- Do not store the device in a wet, humid or dusty location.

If storage conditions are different from the usage conditions indicated in this document, please wait 30 minutes before taking a measurement. While stored, the device should not be exposed to direct sunlight.

- Ensure that you place the protective cap on the sensor after each use to prevent damage to the sensor.

- If you are not using the device for an extended period, remove the alkaline batteries from the device.

Operating & Storage Conditions

Operating conditions of use (Ambient Temperature / Humidity): 10°C -40°C (50°F - 104°F) / RH <= 95%

Storage conditions (Temperature / Humidity / Air Pressure): -25°C to 55°C (-13°F to 131°F) / RH <= 95%

Specification & Technical Data

Product Description: Infrared Temporal thermometer

Model: SCT01

Sensor: Thermopile

Temperature Display: 3 digits (°C) and 4 digits (°F)

Resolution: 0.1°C/0.1°F

Measuring range: 35°C - 43.2°C (95°F - 109.8°F)

Clinical Accuracy: ± 0.2°C on 35.5°C - 42°C range (± 0.4°F on 95.9°F - 107.6°F range) ± 0.3°C (0.5°F) outside this range.

ASTM laboratory accuracy requirements in the display range of 37 to 39 °C (98 to 102 °F) for IR thermometers is ± 0.2 °C (± 0.4 °F), whereas for mercury in- glass and electronic thermometers, the requirement per ASTM Standards E 667-86 and E 1112-86 is ± 0.1 °C (± 0.2 °F).

Power Supply: 2 x 1.5V (LR03 AAA Battery)

Battery life: 2 to 3 years

Automatic switch-off: 30 seconds

Protection against electric shock: Internally Powered ME equipment

Size: L= 116 mm (4.57 in.) x Diam = 33.2 mm (1.31 in.)

Weight: Approx. 75g (battery included) - 0.165 lbs.

Package content: Main unit, 2 AAA (LR03) batteries (already installed), Protective lens cap, Quick start guide, Product guide.

Product Life: 5 years

IP Protection Level: IP 22

Type BF applied part (plastic body)

- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC RF Radiation Exposure Statement

This product emits radio frequency energy, but the radiated output power of this device is far below the FCC radio frequency exposure limits. This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. Nevertheless, end users must follow the specific operating instructions for satisfying RF exposure compliance. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Safety & Performance

Declaration - Electromagnetic Emissions and Immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.

EUROPE - EU DECLARATION OF CONFORMITY

Withings hereby declares that the device Withings Thermo is in conformity with the essential and relevant requirements of applicable EU Directives and Regulations. The full text of the EU declaration of conformity is available at: withings.com/compliance

Federal Communications Commission (FCC) Statement: Model SCT01 FCC ID: XNASCT01

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation of the device.

FCC Statement

\$15.21: You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user's authority to operate the equipment.

\$15.105(b): This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

RF Statement

Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the following. | Portable and mobile RF communication equipment can affect Medical Electrical Equipment.

Guidance and manufacturer's declaration - electromagnetic emissions

The Withings Thermo is intended for use in the electromagnetic environment specified below. The customer or the user of the Withings Thermo should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
CE emissions CISPR11	Group 1	The Withings Thermo uses RF energy only for its internal function. Therefore, its RF emissions are very low and are unlikely to cause any interference with nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	This Withings Thermo is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration - electromagnetic immunity

The Withings Thermo is intended for use in the electromagnetic environment specified below. The customer or the user of the Withings Thermo should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RFIEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the EQUIPMENT or SYSTEM including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol 
Electrostatic discharge (ESD) IEC 61000-4-2	6kV contact 8kV air	6kV Contact 8kV Air	Floors should be wood, concrete or ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	2kV for power supply lines 1kV for input/output lines	N/A	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	1 kV differential mode 2 kV common mode	N/A	A main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	-5% UT (95% dip in UT) for 0.5 cycle, -40% UT (60% dip in UT) for 5 cycles, - 70% UT (30% dip in UT) for 25 cycles, -5% UT (95% dip in UT) for 5 sec	N/A	Mains power quality should be that of a typical commercial or hospital environment. If the user of the EQUIPMENT or SYSTEM requires continued operation during power mains interruptions, it is recommended that the EQUIPMENT or SYSTEM be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	N/A	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or of hospital environment

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Model: SCT01 IC ID: 11411A-SCT01

EN: This device meets the IC requirement for RF exposure in public or uncontrolled environments. This device complies with Industry Canada licence exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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AAA Cells & Disposal

AAA alkaline cells usage

- If AAA alkaline cells fluid should get on your skin or clothing, immediately rinse abundantly with clean water.
- Use only two AAA alkaline cells with this thermometer. Do not use any other type of batteries.
- Do not insert AAA alkaline cells with their polarities incorrectly aligned.
- Replace old AAA alkaline cells with new ones immediately.
- Replace all two AAA alkaline cells at the same time.

Replacing the AAA alkaline cells

If the low battery symbol appears in the Withings application, replace all two AAA alkaline cells at the same time.

1. Remove the battery cover at the back of the thermometer.
2. Install or replace two AAA alkaline cells so that the + (positive) and - (negative) polarities match the polarities indicated on the AAA alkaline cells compartment.
3. Put the battery cover back in place. If the device will not be used for a long period of time, it is advised to remove the alkaline cells.

Disposal

Actuation of European directive 2012/19/EU, for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal. The symbol applied on the device or its packaging means that at the end of its useful life, the product must not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health. In case of abusive disposal of devices by the user, will be applied administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

Warranty

Withings Two (2) Years Limited Warranty - Withings SCT01 | Thermo

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, (“Withings”) warrants the Withings-branded hardware product (“Withings Product”) against defects in materials and workmanship when used normally in accordance with Withings’ published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser (“Warranty Period”).

Withings’ published guidelines include, but are not limited to, information contained in technical specifications, safety instructions or Quick Start Guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free.

Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product’s use.

AU: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage.

You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since you will be storing personal health information. Users will also receive additional update notifications on the device via the app, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes.

Do not install the device on a smartphone that you do not own.

Do not use a public Wi-Fi network you don't know. Use a trusted Wi-Fi network with your device.

The Withings application is not intended to be used on a computer. No indication of anti-virus software is needed.

Only use official app stores to download the Withings application. In case of doubt, use the link go.withings.com.

Equipment Symbol Description

The Withings Thermo complies with the essential requirements of the Directive 93/42/CEE

Medical Device Manufacturer

IP22

Ingress of water or particulate matter

Complies with FCC Regulation

Storage temperature

Complies with waste electrical and electronic equipment directive

Type BF Applied Part (Plastic body)

UK Responsible Person

Pure Offices Plato Close
Warwick CV34 6WE England United Kingdom

Australian sponsor

Emergo Australia Level 20, tower II, Darling Park, 201 Sussex Street, Sydney, NSW 2000, Australia

Read this manual before use

Regulatory Compliance Mark

Caution

European Authorized Representative REPRESENTATIVE

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© 2023 Withings. All rights reserved.