THOMSON

Cardio W7

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WRIST BLOOD PRESSURE MONITOR

TENSIÓMETRO DE MUÑECA

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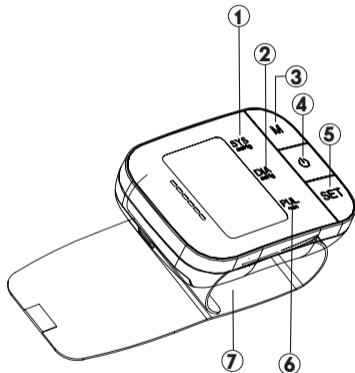
User's guide

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
Modèle # DBP-2170

Description de l'appareil
Device description
Descripción del dispositivo

Afficheur LCD
Display
Pantalla

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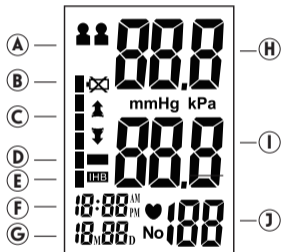INTRODUCTION

Thank you for choosing the THOMSON Cardio W7 Wrist Blood Pressure Monitor. Designed for adults and adolescents over 12, this durable device provides accurate measurements of systolic and diastolic blood pressure, as well as heart rate. For optimal results, position your wrist at heart level during measurement to avoid pressure distortion. Blood pressure fluctuates based on various factors. Home readings may vary from those taken in a medical setting. The WHO classification chart on page 3 can help you understand your results.

We hope you find this user manual helpful and that your new THOMSON Cardio W7 brings you satisfaction and convenience in monitoring your blood pressure.

PRECAUTIONS FOR USE

Measurements of your blood pressure and heart rate are for informational purposes only. Do not self-diagnose or modify your treatment without medical advice. Use the device in accordance with the user manual. The manufacturer declines all responsibility in case of improper or non-compliant use.

PRODUCT COMPOSITION

1 Cardio W7 wrist blood pressure monitor + 1 user manual + 1 storage case

Device Description :

Refer to page 2

Legends of the blood pressure monitor

- | | |
|---|-------------------------|
| 1 - Systolic blood pressure | 5 - «SET» button |
| 2 - Diastolic blood pressure | 6 - Heart rate |
| 3 - «M» button | 7 - Wrist cuff |
| 4 - «  » button | |

Legends of the display screen

A - Users

B - Low battery indicator

C - WHO blood pressure classification indicator

D - Average of the last three irregular results

E - Irregular heart rate indicator

F - Time

G - Date

H - Systolic blood pressure

I - Diastolic blood pressure

J - Heart rate

PRODUCT INFORMATION

- **Certification/Regulatory Information:** complies with the requirements of the applicable European and national directives according to the Medical Devices Regulation (EU) 2017/745.

- **User Manual Information:** designed for simple and intuitive use, this manual will help you take full advantage of all features. Please read it carefully before use and keep it for future reference or for use by other users.

- **Safety Information before and during use of the device:**

- **WARNING:** hazards that can cause serious injury or death.

- **CAUTION:** hazards that can cause minor injury, or damage to the device or accessories.

Symbols used in the user manual, on the packaging or on the label on the back of the device: Some represent the standards and regulatory requirements associated with the device and its use.

CONTRAINDICATIONS

DO NOT USE THIS DEVICE:

- **On newborns or children.**
- **On individuals with implanted cardiac pacemakers.**
- **On individuals with metal implants.**
- **On individuals who have had a mastectomy.**

Do not position the cuff over wounds, as this may worsen them. Do not use the cuff on a wrist where the arteries or veins are subject to treatment or an intravascular device.

DEVICE USAGE

Quick Start

1. Install batteries. (See Figure A)
2. Remove any clothing from your wrist. (See Figure B)
3. Rest for several minutes before testing. Wrap the cuff around your left wrist. (See Figure C)
4. Sit comfortably and place your wrist at heart level. (See Figure D)
5. Press the «  » button to start the test. (See Figure E)

Figure A

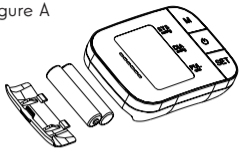
Figure B

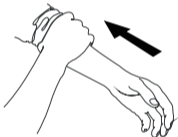
Figure C


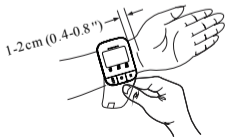
Figure D

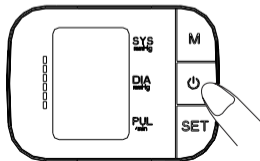Figure E

Battery Installation

Use 2 new AAA alkaline batteries. Observe polarity. Close the compartment. Replace them when the low battery icon appears. Remove them for long periods of non-use.

GENERAL GUIDELINES

Find a quiet place and make sure you have been relaxed for at least 5 minutes. Avoid physical activity, heavy meals, and baths in the half hour before measuring. Sit comfortably **with your wrist supported at heart level**. Stay away from electronic devices.

For better reliability, take measurements at the same time each day and on the same arm. Maintain a stable position and avoid talking or moving. Wait at least 3 minutes between each measurement.

Precautions: This device is not suitable for people with severe arrhythmias. Do not use if it is damaged. Place the cuff directly on the skin and fasten it firmly.

DEVICE SETUP

Time/date setting: With the device turned off, press « SET » twice to access the time/date setting. Use the « M » button to adjust the month, then press « SET » to confirm. Repeat for the day, hour, and minutes.

Saving settings: Press the «  » button to turn off the device. Settings are automatically saved every 3 minutes of inactivity.

TEST

Press the « SET » button to access system settings. The memory group icon - user 1 or user 2 - flashes (position A).

User selection: to determine which user the measurement will be stored for, briefly press the « M » button. You'll see the user via the icon (position A) on the LCD screen, which will flash. To save the selected user, press the «  » button. Each group can store up to 60 measurement results.

Power on: press and hold the «  » button. Wait for the beep.

Note: The device will not operate if there is residual air from a previous test in the cuff. The LCD screen will flash «  » until the pressure stabilizes.

Inflation: the device will automatically inflate to the appropriate pressure. Remain calm and still.

Test: After the cuff is inflated, the air will slowly decrease. A flashing  will appear simultaneously on the screen, indicating the detection of heartbeats. Remain relaxed during the test. Take multiple measurements for greater accuracy.

Displaying results: the screen will display systolic and diastolic blood pressure measurements with a voice announcement. In position « C », an indicator

representing the current measurement will appear next to the corresponding WHO classification.

Irregular heart rate indicator: the symbol **IHB** indicates an irregular rhythm. Consult your doctor if this occurs frequently.

Test result storage: : results are automatically saved. You can delete the last result if necessary. Press the « SET » button after the result is displayed. If the number of tests exceeds the 120 allocated memories, the most recent tests will appear, eliminating the oldest readings.

In summary: turn on the device, let it do its job, and check the results. If you have any doubts, do not hesitate to consult your doctor.

Power off: the «  » button can be pressed to turn off the device at any time. The device will automatically turn off after about 3 minutes of inactivity.

Caution: if the pressure in the cuff becomes too high during the test, press the «  » button to turn off the device.

Memory recall: with the device turned off, you can view previous test results using the «M» button. After activating the test results, you can press the « M » button to scroll through all the test results stored in memory. The LCD screen will display the last measurement result in memory.

Memory deletion: To delete the memory of a selected group, go to Memory Recall mode. Press and hold the « SET » button for about 3 seconds to delete

all records in the selected group. The device will beep. Press the «  » button to turn off the device.

Note: Once deleted, the memory cannot be recovered.

Average of the last 3 tests : with the device turned off, press the « M » button to turn on the display. The screen will show the average results of the last 3 readings from the most recently used group. The **Ave** symbol will appear along with the corresponding WHO blood pressure indicator.

TROUBLESHOOTING

Cleaning and maintenance

Clean the device with a damp cloth (water only). For the cuff, use a soft cloth dampened with rubbing alcohol, then allow to air dry. Clean it before each use by another person. Avoid shocks and exposure to sunlight. Store the device in a dry place at room temperature. Remove the batteries for long-term storage.

Repair and disposal

In case of failure, contact your dealer or after-sales service. Do not attempt to repair the device yourself, as this will void the warranty. First check the condition of the batteries. At the end of its life, do not throw the device away with household waste. Comply with local regulations for electrical and electronic waste (WEEE directive). To report an incident, consult the **EUDAMED platform**, (<https://ec.europa.eu/tools/eudamed/#/screen/home>) indicating the serial number on the rear label of the device.

Battery replacement

Do not dispose of used batteries. Their disposal is regulated. Deposit them at the designated collection points. This is a legal obligation.

TECHNICAL SPECIFICATIONS

Subject to technical changes without prior notice for updating purposes, this blood pressure monitor complies with European regulations and bears the «CE 0197» marking.

It also complies with the following standards (non-exhaustive list):

Safety standards

EN 60601-1 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.

EN 60601-1-2 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic disturbances - Requirements and tests.

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EN 1060-3 Non-invasive sphygmomanometers - Supplementary requirements for electromechanical blood pressure measurement systems.

ISO 81060-2 Non-invasive sphygmomanometers - Part 2: Clinical validation of the automatic measurement type.

Product Description	Automatic Wrist Blood Pressure Monitor Model DBP 2170
Display	Digital LCD: 30.6 mm × 45 mm
Measurement Method	Oscillometric
Measurement Location	Wrist
Automatic Inflation	Yes
Systolic Pressure Range	60 mmHg ~ 260 mmHg
Diastolic Pressure Range	30 mmHg ~ 200 mmHg
Pressure Range	0 mmHg ~ 300 mmHg
Pressure Accuracy	± 3 mmHg
Pulse Range	30 ~ 180 beats/minute
Pulse Accuracy	± 5%
Memory	2 × 120 measurements with date and time
Power Supply	2 x AAA alkaline batteries 1.5V (not included)
Automatic Shut-off	After 3 minutes
Weight	Approximately 114g (without batteries)
Dimensions	85 x 62 x 25 mm
Cuff Size	Fits wrists 13.5 to 21.5 cm
Battery Life	2 months with 3 measurements per day
Operating Environment	Temperature: 10-40°C, Humidity: 15%- 93%, Air Pressure: 700hPa- 1060hPa

Storage Environment Temperature: -25°C - 70°C, Humidity ≤ 93% HR

Classification Internal power supply type BF, index P22

2-year warranty. We will repair or replace your blood pressure monitor free of charge in the event of a manufacturing defect. The warranty does not cover damages caused by misuse. Please contact your retailer for more details..

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Electromagnetic Compatibility Information

47

The device satisfies the EMC requirements of the international standard IEC 60601-1-2. The requirements are satisfied under the conditions described in the table below. The device is an electrical medical product and is subject to special precautionary measures with regard to EMC which must be published in the instructions for use. Portable and mobile HF communications equipment can affect the device. Use of the unit in conjunction with non-approved accessories can affect the device negatively and alter the electromagnetic compatibility. The device should not be used directly adjacent to or between other electrical equipment.

Table 1

Guidance and declaration of manufacturer-electromagnetic emissions		
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment -guidance
Radiated emission CISPR 11	Group 1, Class B	The device uses RF energy only for its internal function. Therefore, its emissions are very low and are not likely to cause any interference in nearby electronic equipment.
Conducted emission CISPR 11	N/A	
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	N/A	

Electromagnetic Compatibility Information

48

Table 2

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment -guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	Floors should be wood, concrete or ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrostatic transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for signal/computer lines	N/A	
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	N/A	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	40% UT (-95% dip in UT) for 5 cycle 70% UT (-30% dip in UT) for 2.5 cycle -5% UT (-95% dip in UT) for 5 cycle	N/A	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m; 50Hz or 60Hz	30 A/m; 50Hz or 60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Electromagnetic Compatibility Information 49

Table 3

Guidance and declaration of manufacturer-electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment -guidance
Conducted RF IEC 61000-4-6	3V for 0.15-80 MHz, 6V /m ISM and amateur radio bands between 0.15-80MHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	385MHz, 27V /m	385MHz, 27V /m	Recommended separation distance $d = \left[\frac{3.5}{f} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{7}{f} \right] \sqrt{P}$ 800 MHz to 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
	450MHz, 28V /m	450MHz, 28V /m	
	710MHz, 745 MHz, 780MHz 9V/m	710MHz, 745 MHz, 780MHz 9V/m	
	810MHz, 870 MHz, 930MHz 28V/m	810MHz, 870 MHz, 930MHz 28V/m	
	1720MHz, 1845 MHz, 1970MHz 28V/m	1720MHz, 1845 MHz, 1970MHz 28V/m	
	2450MHz, 28V /m	2450MHz, 28V /m	
5240MHz, 5500 MHz, 5785MHz 9V/m	5240MHz, 5500 MHz, 5785MHz 9V/m		

Electromagnetic Compatibility Information 50

Table 4

Recommended separation distances between portable and mobile RF communications equipment and the device		
The device is intended for use in an electromagnetic environment in which radiated therefore disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.		
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter	
	80 MHz to 800 MHz $d = \left[\frac{3.5}{f} \right] \sqrt{P}$	800 MHz to 2.7 GHz $d = \left[\frac{7}{f} \right] \sqrt{P}$
0.01	0.12	0.23
0.1	0.38	0.73
1	1.2	2.3
10	3.8	7.3
100	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE1 At 80 MHz and 800 MHz, the separation distance for the **higher** frequency range applies.

NOTE2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

JOYTECH Healthcare Co., Ltd.
No. 365, Wuzhou Road, 311100 Hangzhou, province du Zhejiang,
RÉPUBLIQUE POPULAIRE DE CHINE

Shanghai International Holding Corp. GmbH (Europe) Eiffestrasse 80,
20537 Hambourg, Allemagne

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