*emmy*  
by Fizimed

## USER GUIDE

<https://app.fizimed.com/video>

**P. 03** EN / ENGLISH

**P. 06** FR / FRANÇAIS

**P. 09** DE / DEUTSCH

**P. 12** ES / ESPAÑOL

**P. 15** SV / SVENSKA

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# EN

## Documentation scope

Please read this manual carefully before using the product. This user manual explains how to use the "Emy - Kegel exercises" mobile application and the Emy connected device. It covers:

- Versions 9.0.0 and higher of the Emy mobile application,
- Versions 7.0.0.0 and higher of the Emy connected device.

If in doubt about the version of your Emy application or device, contact the manufacturer, Fizimed, who will provide you with the appropriate documentation.

The application supports iOS14, Android 6.0 or higher.

## Product overview

If upon the receipt of the product, you notice that the sleeve is unsealed, torn or damaged, do not use the product and contact the seller.

Part of the device-

1. Intravaginal part; sensors
2. Operating indicator light
3. Charge indicator light; charging magnets; Bluetooth
4. Storage case
5. USB magnet cable

Emy is a device for intravaginal use to strengthen the perineum at home.

Size: 105 x 34 mm

Weight: 55 g

## Cleaning

Clean the Emy device with mild soap and water before and after each use. If you are susceptible to infections, use a pH-neutral antibacterial cleanser for intimate accessories. Make sure the device is completely dry before storing it in its case. The Emy case should be cleaned with a disinfectant wipe.

## Indicator light operation

### Operating indicator light (2)

• Repeated oscillations: the device is switched on and trying to connect to the mobile application.

- 1 short flash every 3 seconds: the device is connected to the mobile application.
- 3 rapid flashes: device is discharged.

• Off: the device is switched off.

### Charge indicator light (3)

This LED will only light up if the device is connected.

- Permanently lit: the device is charging.
- Off: device is 100% charged.

## Battery recharge

Place the magnetized part of the supplied cable (5) over the recharging magnets (3) and

connect the USB plug to an AC adapter or computer. The more you use your device, the longer it will last. An inactive battery increases the risk of premature wear and reduces its lifespan. If you don't use your device regularly, be sure to recharge it for 2 hours every three months.

## Use

 Download "Emy - Kegel exercises" on the **App Store** or **Google Play**. Let it guide you to create your account and choose your training program!

• Activate your phone's Bluetooth and localization.

• Please do not pair the device with your phone in the Bluetooth settings. Emy automatically connects to the application without prior association.

• On the mobile application, start a session. Select a session with the device, then continue through the following screens.

• Please click on "Authorize" for each authorization request (location, Bluetooth, nearby devices and sharing).

• Before clicking on "I'm ready", shake the device to switch it on. The indicator light (2) will oscillate slowly: the device could take up to 2 minutes to connect to the application.

• Then insert the inner part of the device (up to the hollow) into the vagina. If you have difficulty inserting the device, you can apply a water-based lubricant. Caution: do not use silicone- or oil-based lubricants, as they may damage the silicone.

• Once the device is switched on and inserted, click on "I'm ready" and the application will quickly start an exercise session.

• The recommended position for Emy is semi-recumbent. Lie down, elevate your neck (with a cushion, for example) and spread your thighs slightly apart. Don't forget to breathe deeply during your exercises and relax all your other muscles (abdominal, buttocks...).

• Several game universes are available. Choose one and perform your exercises, following the contraction and release instructions displayed on screen.

• Make sure you only contract your pelvic floor. Your stomach, thighs and buttocks should be relaxed. Inhale as you release the contraction, then exhale as you contract your perineum.

• Be careful not to cover the white part of the device. To maintain a stable connection, keep your smartphone within 1 meter of the device. If a disconnection occurs during a workout, the workout will pause and the device will automatically reconnect.

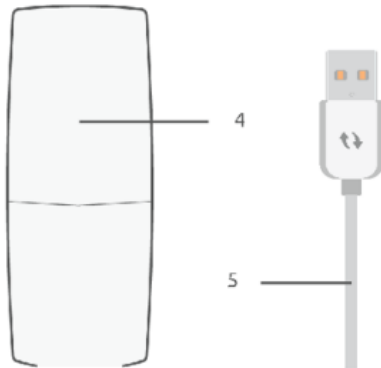
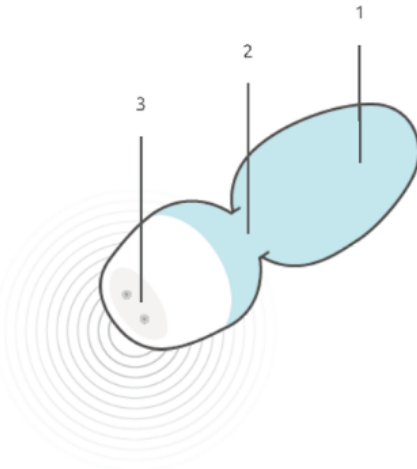
If after 2 months there are no results, please consult a healthcare professional.

## Safety instructions

Do not leave the device in the vagina for more than 30 minutes.

Do not microwave.

Do not drop the Emy device.

Remove device before using the toilet.  
Do not use the device if it is damaged or altered in any way.  
Do not pierce, cut or slice the device.  
Do not clean the device with chemical or abrasive products.  
Do not put device in dishwasher.  
Do not sterilize the device in boiling water.  
Keep device away from heat sources.  
Keep out of reach of children and pets.  
Do not use the device in a bath or shower.  
Do not lend or share the device with anyone.  
Do not insert the device into any part of the body other than the vagina (intravaginal device only).  
Wash the outer part of the device outside the vagina.  
Do not have sexual intercourse while the device is inserted.

Any serious incident involving the device must be reported to Fized and to the competent authority of the member state in which the user is established.

### Contraindications

If you are pregnant or have given birth less than 7 weeks ago.  
If you experience constant pain when using or inserting the device.  
If you currently have health problems related to your pelvic floor.  
If you suffer from gynaecourinary cancer.  
If you suffer from extra-urethral incontinence (fistula, ectopic ureter).  
If you suffer from severe urinary retention.  
If you have any doubts about your ability to use this device, please consult a healthcare professional.

### Warranty

The legal warranty on the Emy device covers all manufacturing and operating defects for 2 years from the date of purchase shown on the invoice, subject to the following two conditions:

- Communication of device serial number,
- Use in accordance with instructions.

The warranty does not cover:

- Damage due to abnormal use or use outside the scope of the originally intended use,
- Damage caused by negligence or accident,
- Damage caused by improper cleaning not in accordance with the instructions,
- Damage caused by an electrical storm,
- Unauthorized maintenance, attempts to open or repair the device.

To trigger the warranty, contact your vendor's customer service department, who will register your claim.

### Service life

The lifetime of a medical device is a mandatory piece of information whose purpose is to estimate the point at which a product becomes obsolete. After this period, the product may no longer function properly, or may represent a risk. Product life may vary according to

maintenance, conditions of use and storage. Fized estimates this life to be 5 years if all conditions of use, storage and cleaning have been respected. If the product is damaged or compromised in any way, it is imperative to end-of-life the device.

 The Emy device must not be disposed of with general household waste. It is your responsibility to dispose of this item by taking it to an authorized collection point for the recycling of electrical and electronic equipment. Consult your local authorities for the location of these facilities. Please note that the device's lithium battery is a fire hazard and may explode if incinerated.

### Product marking

This medical device is a regulated healthcare product and, in accordance with these regulations, bears the CE mark. The Emy device is a medical device designed to offer women a pelvic floor re-education method combining a device and a mobile application to improve their quality of life.

### FCC

All products sold in countries requiring FCC certification are FCC compliant. This device complies with Part 15 of the FCC rules. Its operation is subject to the following two conditions:

- This device may not cause harmful interference, and
- This device must accept any interference received, including interference that may cause undesired operation.

### FCC RF Exposure Statement

This equipment complies with FCC radiation exposure limits established for an uncontrolled environment. End-users must follow specific operating instructions to meet RF exposure compliance. This transmitter complies with the limits for portable and mobile devices as demonstrated by the RF radiation exposure analysis and SAR test report. This transmitter must not be co-located or operated in conjunction with any other antenna or transmitter, except in accordance with FCC procedures for multi-transmitter products.

### Manufacturer's responsibility to comply with FCC regulations

The transmitter module must not be co-located or operated in conjunction with any other antenna or transmitter, except in accordance with FCC procedures for multi-transmitter products.

EMY  
EMY contains:  
A transmitter module:  
FCC ID: QOQ-BGM220S2

### Protection class

The device is IP67: fully protected against dust. Protected against the effects of immersion to a depth of 1 m.

### Storage, transport and use conditions

When using the Emy device, the following operating conditions must be observed:

- Operating temperature: 0°C to 38°C,
- Relative humidity range from 10% to 95%, non-condensing, but not requiring a partial water vapour pressure greater than 50 hPa,
- Atmospheric pressure range from 700 hPa to 1060 hPa.

Transport and storage conditions between uses are as follows:

- -20°C to +70°C without relative humidity control,
- +5°C to +35°C with relative humidity up to 95%, non-condensing,
- Atmospheric pressure range from 700 hPa to 1060 hPa.

### Emy device electronic features

The Emy device is designed for home use. It is an internally powered medical device. The product contains a rechargeable Lithium Ion battery. The product is rechargeable only via the cable supplied. The device supply current is: DC 5V / 0.5A.

The device is approved to the EN 60601-1-2 electromagnetic compatibility standard for medical equipment. As such, the Emy device must be used in accordance with the information and instructions provided in this manual.

This electromedical device contains an internal power supply.

The applied part is type BF (IEC 60601-1).

### EMC information

- RF emissions from the Emy device are very low and are not likely to cause interference in nearby electronic equipment,
- The Emy device is suitable for use in domestic premises,
- The Emy device uses wireless communications in the following frequency bands: Bluetooth BLE: [2.400GHz -2.485GHz],
- The Emy device complies with the limits of ETSI EN 300 328 V2.1.1,
- The Emy device complies with the limits of standard EN 55011 (2010),
- The Emy device complies with the limits of EN 61000-6-3 (2007 +A1/2011),
- The Emy device complies with the limits of IEC 60601-1-2 Ed.4.0 (2014) and EN 61000-4-3 and EN 61000-4-8 for immunity to magnetic fields,
- The Emy device complies with the Class B requirements of ETSI EN 301 489-1, ETSI EN 301 489-17 and EN 55022 for radiated emissions,

- The Emy device complies with the requirements of ETSI EN 301 489-1, ETSI EN 301 489-17 and EN 61000-4-2 for electrostatic discharge.
- The Emy device meets the requirements of ETSI EN 301 489-1, ETSI EN 301 489-17 and EN 61000-4-3 for immunity to radiated electromagnetic fields (see Appendix 1).
- The Emy device complies with the limits of IEC 60601-1-2 Ed.4.0 and EN 61000-4-2 with regard to immunity to electrostatic discharge (see Appendix 1).

All EMC results tables are available on request from the Manufacturer.

### Clinical benefits of the Emy device

The clinical benefit of the Emy device is improved quality of life for patients.

### Emy device performance

The Emy device is sensitive enough to detect very weak contractions of the pelvic floor muscles, so it can be used for postpartum rehabilitation or in cases where the perineal muscles have lost their tone.

### Glossary of symbols

	CE marking		Attention
	Manufacturer		Moisture resistant
	Electromagnetic field Non-ionizing radiation		The waste must be collected by a specific channel and must not be disposed of in the usual garbage can.
	Serial number		Temperature limit
	Unique device identifier		Humidity limit
	Catalog number		Atmospheric pressure limit
	Type BF protection against electric shock		Direct current
	It is essential to consult the precautions for use.		Medical devices
	Refer to operating instructions		Class II equipment
	Made in Europe		Fragile: handle with care
	One patient, many use		Date of manufacture
	Swiss agent		

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Fizimed SAS
CMC Medical Devices GmbH