Artikelnummer: Z 00926 (Modellnummer BE201)

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Article number: Z 00926 (model number BE201)

Dear Customer,

We are delighted that you have chosen this **back support belt**. Before using the article for the first time, please read the instructions through carefully and keep them for future reference. If the article is given to someone else, it should always be accompanied by these instructions. If you have any questions about the article, contact the customer service department via our website:

[www.service-shopping.de](http://www.service-shopping.de)

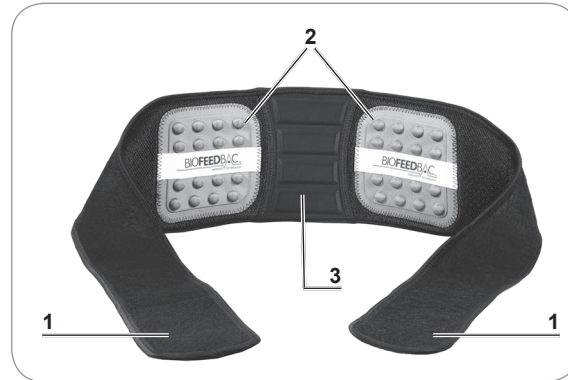
### IMPORTANT INSTRUCTIONS

- The belt is designed to be used to relieve stress on the lower back muscles.
- Suitable for a waist measurement of approx. 55-140 cm.
- The article is not suitable for therapeutic purposes.
- People with a slipped or bulging disc, degenerative joint disease, disorders of the spinal column, lumbar injuries, fractures of the spine, pelvis or ribs, osteoporosis and also serious cardiovascular or respiratory diseases may only use the article following consultation with a doctor.
- Pregnant women should not use the article.
- If you experience any pain or irritation while wearing the belt, take it off and try it again after 24 hours. If the problems persist, consult a doctor before you put the belt on again.
- The article is designed for personal use only and is not intended for commercial applications. Use the article only as described in the instructions. Any other use is deemed to be improper.
- Keep the packaging material away from children and animals! There is a danger of suffocation from plastic films.
- The article is **not** a toy! Keep children and animals away from the article in order to prevent injuries.
- Only wear the article in the area of the abdomen and back. The article is not suitable for use on other parts of the body. The article should never be placed around the neck.
- Do not use the article if it is damaged!
- Do not expose the article to extreme temperatures, wide temperature variations, moisture, wet conditions or direct sunlight. Keep the article away from fire (e.g. burning candles) and heat sources.
- Make sure the article does not come into contact with cleaning products or chemicals.
- If the article is dirty, wipe it with a damp cloth. It must not be washed in the washing machine or immersed in water.
- Keep the article dry and cool.
- The warranty does not cover any defects which are caused by failure to comply with these instructions and the safety and care instructions, improper handling or damage. This also applies to normal wear and tear.

### Items Supplied

- 1 x back support belt
- These instructions

### Article Overview

- 1 Velcro fastener
- 2 Pressure point pads
- 3 Support cushion

### Use

1. You can wear the belt directly on your skin or over a thin top, e.g. a vest or tight-fitting T-shirt. If you have a thicker top on, lift it up to put on the belt. You can keep trousers or a skirt on.
2. Place the belt around your lower back. The pads must touch your back when you do this. The logo on the two pressure point pads should be the right way round and not upside down.
3. Align the belt so that the support cushion is on your spine.
4. Do up the Velcro fastener around your stomach. The belt must not be too loose or too tight. Correct the width if necessary until the belt fits comfortably.
5. We recommend that you wear the belt for 30 minutes at first to get used to it. You can then increase the length of time you wear it to reflect your needs.

### Disposal

The packaging material is recyclable. Please dispose of it in an environmentally friendly manner and take it to a recycling centre.

Customer Service / Importer:

DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Germany • ☎ +49 38851 314650 \*)

\*) Calls to German landlines are subject to charges. The cost varies depending on the service provider.

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All rights reserved.
DS Produkte GmbH, Am Heisterbusch 1, 19258 Gallin, Duitsland • ☎ +49 38851 314650 \*)