beurer

SL 70

- | | |
|--|--|
| (DE) Schnarchstopper
<i>Gebrauchsanweisung</i> 2 | (IT) Dispositivo antirussamento
<i>Istruzioni per l'uso</i> 35 |
| (GB) Snore stopper
<i>Instructions for use</i> 11 | (TR) Horlama durdurucusu
<i>Kullanım Kılavuzu</i> 43 |
| (FR) Oreillette anti-ronflements
<i>Mode d'emploi</i> 19 | (RUS) Устройство против храпа
<i>Инструкция по применению</i> 51 |
| (ES) Dispositivo antirronquidos
<i>Instrucciones de uso</i> 27 | (PL) Urządzenie zapobiegające chrapaniu
<i>Instrukcja obsługi</i> 60 |

DEUTSCH

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Beurer GmbH
Servicecenter
Lessingstraße 10b
89231 Neu-Ulm
Germany

ENGLISH

Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Table of contents

1. Getting to know your device	12	5.3 Use	16
2. Signs and symbols	13	6. Charging the battery	16
3. Notes	13	7. Transferring data to your smartphone	17
4. Unit description	14	8. Disposal	17
5. Initial use	15	9. Technical data	17
5.1 Connecting the snore stopper to the "beurer SleepQuiet" app	15		
5.2 Attaching the snore stopper	15		

Dear customer,


Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications related to heating, weight, blood pressure, body temperature, pulse, gentle therapy, massage, air and beauty.

With kind regards,
Your beurer team

Included in delivery

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

1x snore stopper
3x earplugs (small, medium, large)
1x micro USB charging cable
1x mains adapter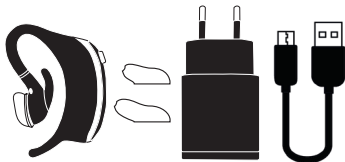


1. Getting to know your device

A restorative night's sleep is of huge importance for our physical and psychological well-being; after all, it is during sleep that we process things such as new experiences and improve our mental and physical fitness. If you snore during your sleep, however, then the quality of your sleep is impaired considerably. The beurer SL 70 snore stopper was developed to reduce snoring, and to increase sleep quality as well as your well-being, for a restful night.

How does the snore stopper work?

The snore stopper is attached to the left or right ear. The snore stopper checks for snoring noises during use. In addition, snoring produces vibrations which are transmitted via the cranial bones and which are also detected by the snore stopper. As soon as the snore stopper detects noises and vibrations, it starts sending impulses to the ear. These impulses consist of tone or tone and vibration. The intensity of these impulses increases from a low level until the desired success is achieved. The throat muscles become firm and the airways open as a reaction to the impulses. Breathing is calmed and the snoring decreases. You may also change your lying position. In the long term, there is a lasting reduction in snoring through the training of the muscles in the throat. Initially, it may take up to 7 days until you are accustomed to the device.

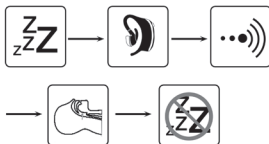The “beurer SleepQuiet” app provides the following functions:

- Detailed snoring analysis including snores per hour, recording duration and determination of the snoring volume
- Calendar with snoring diary
- Evaluation view (in days/weeks/year)
- Adjustable impulse intensity and sensitivity

System requirements:

iOS from version 8.0, Android™ from version 4.4 with *Bluetooth®* 4.0

List of supported devices:

2. Signs and symbols

The following symbols appear in these instructions for use:

	WARNING Warning instruction indicating a risk of injury or damage to health
	IMPORTANT Safety note indicating possible damage to the device/accessory
	Note Note on important information
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Manufacturer
	This product satisfies the requirements of the applicable European and national directives.

3. Notes

WARNING

- Protect the snore stopper from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Keep packaging material and the snore stopper away from children. Choking hazard!
- Keep the snore stopper away from heat sources and do not use it near (~1 m) shortwave or microwave devices.
- Use and store the snore stopper at an ambient temperature between 0°C and 60°C. Higher or lower temperatures may damage the snore stopper.
- The snore stopper is only splash-proof.
- Keep the snore stopper away from water.
- Do not use any chemical or abrasive cleaning agents. Clean the snore stopper occasionally with a 70% isopropyl alcohol pad.
- If your skin or eyes come into contact with fluid from a rechargeable battery cell, flush out the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.

- Risk of explosion! Never throw batteries into a fire.
- Do not disassemble, split or crush the rechargeable batteries.
- Fully charge the battery prior to initial use.
- Fully charge the battery at least every 3 months.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.

4. Unit description

1 ON/OFF button

2 Operation light

Green = battery at 41–100%

Red = battery at 0–40%

3 *Bluetooth*[®] light

4 Micro USB connection

5 Earplugs

5. Initial use

5.1 Connecting the snore stopper to the “beurer SleepQuiet” app

You can also use the basic functions of the snore stopper without the “beurer SleepQuiet” app. The additional features such as recording, statistics, calendar and settings are only available in conjunction with the “beurer SleepQuiet” app.

To connect your snore stopper with the “beurer SleepQuiet” app, please proceed as follows:

1. Ensure that your smartphone is connected to the internet.
2. Activate *Bluetooth*® on your smartphone.
3. Download the free “beurer SleepQuiet” app from the Apple App Store (iOS) or at Google Play (Android).

Depending on the settings for your smartphone, additional charges may be incurred for connecting to the Internet or transferring data.

4. Start the “beurer SleepQuiet” app and follow the instructions.


5.2 Attaching the snore stopper

You can attach the snore stopper to the left or right ear.

Here, it is important to turn the earplug [5] so that the opening is facing the direction of the ear canal once attached.

- Attach the snore stopper to your ear as shown. Do **not** bend the upper clip on the snore stopper too strongly as it may become damaged.

1.

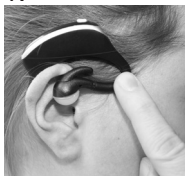2.

3.

If the earplug easily moves in your ear, try one of the other earplugs included in delivery for optimum fit.

5.3 Use

1. Press and hold the ON/OFF button on the snore stopper for three seconds. The operation light begins to light up. The snore stopper is now switched on. If the snore stopper cannot be switched on, the battery in it is flat. Re-charge the battery as described in Chapter “6. Charging the battery”. After a short time, the operation light switches off so that it does not disturb you in the night.
 2. Attach the snore stopper to your left or right ear as shown.
 3. Go to bed.
 4. As soon as you get up, switch off the snore stopper. To do this, press and hold the ON/OFF button on the snore stopper for three seconds. A short vibration confirms that the snore stopper has been successfully switched off.
- Initially, it may take up to 7 days until you are accustomed to the device.

6. Charging the battery

1. Open the cover on the micro USB connection of the snore stopper.
2. Connect the charging cable included in delivery to the snore stopper and a computer that is switched on. Alternatively, connect the charging cable with the snore stopper and the mains adapter. Insert the mains adapter into a suitable socket.
3. Whilst charging, the operation light flashes green. As soon as the battery is fully charged, the operation light will remain green.
4. It usually takes approx. 3 hours to fully charge the battery. As soon as the battery is completely charged, remove the charging cable from the computer or the socket and the mains adapter and the snore stopper.
5. When the battery is fully charged, the battery life is approx. 4–5 days. The operation light turns red as soon as the battery capacity is below 41%.

7. Transferring data to your smartphone

To transfer the recorded data from your snore stopper to the “beurer SleepQuiet” app, proceed as follows:

1. Activate *Bluetooth®* on your smartphone.
2. Open the “beurer SleepQuiet” app. Press the ON/OFF button on the snore stopper. The recorded data is now transferred to the smartphone. For a large volume of data, this may take some time.

8. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

9. Technical data

Dimensions	49 x 15 x 62 mm
Weight	15 g (without earplugs)
Connection	Micro USB connection
Mains adapter voltage supply	Input: 100-240 V~; 50/60 Hz; 0.3 A Output: 5 V $\overline{\text{---}}$ 1000 mA
Mains adapter protection class	 double-protected
Battery	80 mAh lithium polymer
Battery life	Approx. 4 days (depending on use)
Memory	1 night for heavy snoring
Material	Silicone

Data transfer	<i>Bluetooth®</i> low energy technology
Permissible operating conditions/storage conditions	0°C to 60°C, 30 to 75 % RH 860 – 1060 hPa ambient pressure
	-20°C to 60°C, <75 % RH 860 – 1060 hPa ambient pressure

We hereby guarantee that this product complies with the European R&TTE Directive 1999/5/EC. Please contact the specified service address to obtain more detailed information such as the CE conformity declaration.

The *Bluetooth®* word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Beurer GmbH is under license. Other trademarks and trade names are those of their respective owners.

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