BPM65

Wrist Type Blood Pressure Monitor

PLEASE READ ALL THE INSTRUCTIONS BEFORE USE

duronicon.com

[duronicon](https://www.youtube.com/channel/UCduronicon)

[@duroniconmedia](https://www.instagram.com/duroniconmedia)

[Duronicon](https://www.facebook.com/duronicon)

[@duroniconmedia](https://www.instagram.com/duroniconmedia)

Contents

English Manual	3
Manuel d'utilisation en français	22
Gebrauchsanleitung auf Deutsch	38
Manual de Instrucciones en Español	54
Manuale D'istruzioni In Italiano	70
Instrukcja obsługi. Polski	86
Türkçe Kullanım Kılavuzu	102
Gebruikershandleiding Nederlands	118
Svensk manual	134

English Manual

Introduction


Thank you for purchasing the Duronic BPM65 Wrist Type Blood Pressure Monitor. This Manual is written and compiled in accordance with Medical Device Regulation (MDR) 2017/745 of the European Parliament and of the Council. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

Please read this User Manual carefully before using this product and retain for future reference.

Box Contents

- 1 x blood pressure monitor
- 1 wrist cuff
- 2 x AAA batteries
- User Manual

Components

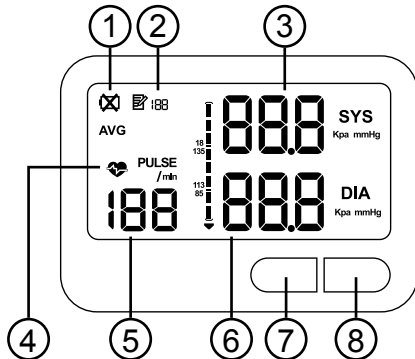- | | |
|------------------------------------|-----------------------------|
| 1. Low battery | 5. Pulse (bpm) |
| 2. Saved measurement number | 6. Diastolic blood pressure |
| 3. Systolic blood pressure | 7. Memory button |
| 4. Heartbeat / Arrhythmia detected | 8. Start/Stop Button |

Safety Information

PLEASE READ THIS MANUAL CAREFULLY AND RETAIN FOR FUTURE REFERENCE

- Advance age as well as certain conditions such as diabetes, hyperlipidemia and atherosclerosis may impact results. It may be more suitable for individuals with these conditions to use an arm blood pressure monitor.
- This device is intended for adult use only. Do not use this device on children under 12 years of age.
- Please keep the product out of reach from children and pets.
- This device is intended for non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on any extremities other than the wrist and should not be used for any other functions other than obtaining a blood pressure measurement.
- This appliance alone should not be a substitute for actual medical care. Seek professional medical advice when necessary.
- Contact your doctor if you are concerned about your blood pressure. This device is for self-monitoring and should not be used for self-diagnosis.
- Do not adjust, begin or end medical treatment or medication without asking a doctor for treatment advice.
- If you are taking medication, consult your doctor to determine the most appropriate time to measure your blood pressure.
- Do not repeatedly measure as it could result in wrist soreness.
- When this device is used to measure patients who have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, the results from this device may vary.
- The device is not suitable for use in the presence of a flammable anaesthetic mixture of air with oxygen or nitrous oxide.
- To avoid measurement errors, please avoid using the device in areas with a strong electromagnetic field radiated interference signal or electrical fast transient/ burst signals.

Safety Information

- Don't use a phone near this appliance as it could result in it malfunctioning.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used. Do not operate the appliance if any part is damaged.
- This device is not suitable for continuous monitoring during medical emergencies or operations.
- Use only Durodic authorised parts and accessories. Parts and accessories not approved for use with the device may result in operational failure.
- If you have any problems with this device, such as setting up, maintaining or using the device, please contact the Durodic Customer Care Team for assistance.
- Do not open or repair the device by yourself. If you experience any problems with your device, contact Durodic for assistance.
- When cleaning the device, please use a soft cloth to clean the whole unit. Do not submerge it in water or any other liquid, and do not use any abrasive or volatile cleaners. Do not use chemical reagents like thinner or alcohol to clean this product.

When using Batteries

- The operator or user of this device should not touch the output of batteries and the patient simultaneously.
- Please ensure that the correct size batteries are used, as described in this manual. Do not use any other type of battery as this may damage the device.
- If any battery fluid should leak and get into your eyes, rinse your eyes with plenty of clean water and contact a doctor immediately.
- Remove batteries from the wrist blood pressure monitor if not in use for a long period of time.

Before You Begin

Batteries

1. Turn the wrist blood pressure monitor upside down and slide the cover off by pushing it off.
2. Put in 2x AAA batteries into the cassette.
3. Clip the cover back into place.

Replace the batteries when the following occurs:

- The 'Low battery' icon shows
- The display dims
- The display does not light up

Caution:

- *Remove batteries if the device is not likely to be used for some time.*
- *Old batteries are harmful to the environment, do not dispose them with other daily rubbish. Remove the old batteries from the device and follow your local recycling guidelines. Do not dispose of batteries in fire; batteries may explode or leak.*

Setting the Unit

When the monitor is off, press and hold down the 'MEMORY' button for 3 seconds. The unit of measurement will change between 'Kpa' and 'mmHg'.

Memory

This blood pressure monitor can automatically store 1×99 groups of measured values and can calculate the average value of recent three groups of measuring results.

Set the right unit information before taking a measurement to ensure the correct information is saved.

Before You Begin

Access the Memory

To access the memory, short-press the 'Memory Button'. "AVG" will show in the top-left corner. This is the average calculated from the latest 3 measurement values saved on the device.

To access individual records short-press the 'Memory Button' again. To cycle through the records keep short-pressing the 'Memory Button'.

Delete the Memory

To delete the memory, short-press the 'Memory Button'. Then press and hold the memory button again for 3 seconds. The screen will show 3 dashes (---) This means the records have all been deleted.

**Note:**

When you save the 100th measurement, the earliest one will be deleted.

Arrhythmia / Irregular Heartbeat Detector


This blood pressure monitor is equipped with Irregular Heartbeat (IHB) Detection. During each measurement this monitor records the heartbeat intervals and works out the standard deviation. If an inconsistent heartbeat is detected, this equipment will light up the 'Arrhythmia' symbol on the screen when displaying the final measured result.

**Caution:**

The appearance of the 'Arrhythmia' icon usually is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

How to Take a Measurement

1. Ensure that the cuff is fitted properly on your arm.
2. Sit in a chair making sure your body is not bent or leaning forward.
3. Before measurement, take a deep breath three or four times, relax your shoulders and arms. Make sure your body is relaxed.
4. Raise the hand with the cuff to be in line with your heart and then stay still. Use a cushion to help keep your arm steady.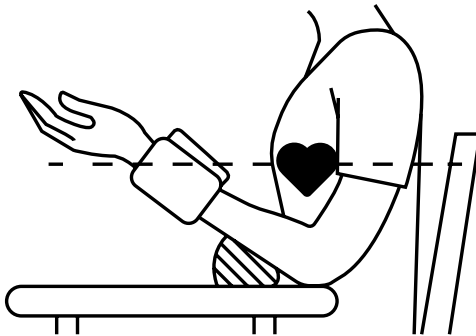

5. Ensuring your hands are extended and open, press the 'Start/Stop Button'. The device will complete the whole measurement automatically. Remain still for the duration of the measurement.
6. The cuff will inflate. When fully inflated the cuff can feel slightly tight or uncomfortable. This is normal. If it is intolerable or you have any concerns, press the 'Start/Stop Button' to stop the machine and seek professional medical advice.
7. The machine will automatically save your results.

Tips for Obtaining an Accurate Measurement

Please follow the below tips to ensure you achieve an accurate measurement:

- Don't hold the wrist cuff when the measurement is being taken.
- Relax your arm and body during measurement.
- Wait at least 1 hour after eating or drinking.
- Do not measure immediately after consuming caffeinated drinks such as tea and coffee, or after smoking.
- Wait at least 20 minutes after taking a bath.
- Do not take a measurement when talking or moving your fingers.
- Do not take a measurement in a very cold environment.
- Do not take a measurement when you need to use the toilet.

Maintenance

In order to get the best performance, please follow the below instructions:

- Store in a dry place and avoid direct sunlight.
- Avoid the monitor touching water. If the machine gets wet, dry immediately afterwards.
- Avoid intense shaking and collision.
- Avoid dusty environments and unstable temperatures.
- Use a wet (well wrung) cloth to remove any dirt.
- Never immerse the cuff in water or any other liquid.

Why does my blood pressure vary throughout the day?

Individual blood pressure varies naturally multiple times during the day. It is also affected by the way you tie your cuff around your wrist and the position of your arm during measurement. For this reason, we advise you to take the measurement at the same time of day and in the same position.

Please note:

- *Medication can affect blood pressure and can make the results vary more.*
- *If you wish to measure a second time, wait at least 3 minutes before taking another measurement.*

Why do I get a different blood pressure at home compared to the hospital?

Blood pressure is different even throughout the day due to a variety of factors like weather, emotions and exercise. Blood pressure usually increases in clinical settings due to anxiety or nerves.

Pay attention to the following when you measure your blood pressure at home:

- Is the cuff is tied properly?
- Is the cuff is too tight or too loose?
- Is the cuff is tied on the upper arm?
- Do you feel anxious?


Advice: Take 2-3 deep breaths before beginning a measurement as this will help get a more accurate result. If you are particular anxious or upset, relax yourself for 4-5 minutes until you calm down.

Which arm should I use to take measurements?

It is fine to use the cuff on either arm, but some people will experience a difference in measurement results.

For this reason, we recommend using the same arm every time.

Understanding the Result

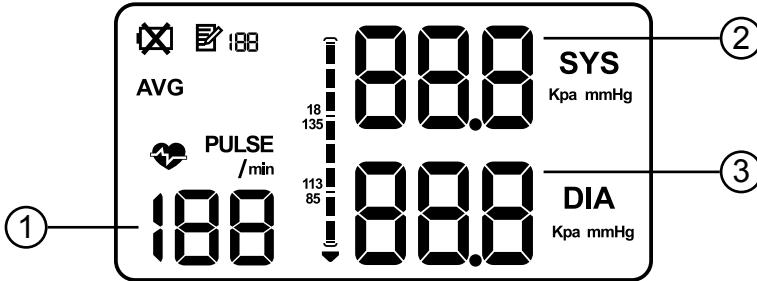1. **Pulse:** Number of heartbeats per minute. Most adults have a resting heart rate between 60 and 100bpm. Athletes are likely to have a lower resting heart rate.
2. **Systolic pressure (SYS):** The highest level of your blood pressure – when your heart beats, it contracts to pump blood through your arteries.
3. **Diastolic pressure (DIA):** The lowest level of your blood pressure – when your heart relaxes between beats.

Standard Blood Pressure Classification

Below illustrates the standard blood pressure classification for adults made by the World Health Organisation (WHO) and International Society of Hypertension (ISH) in 1999.

Caution:

- *Only a doctor can tell you your normal blood pressure range and the point at which you are at risk. Consult your doctor to obtain these values.*
- *If the measurements taken with these products fall outside the range, consult your GP/doctor for advice.*

Understanding the Result

Table 1: Healthy and unhealthy blood pressure chart for adults

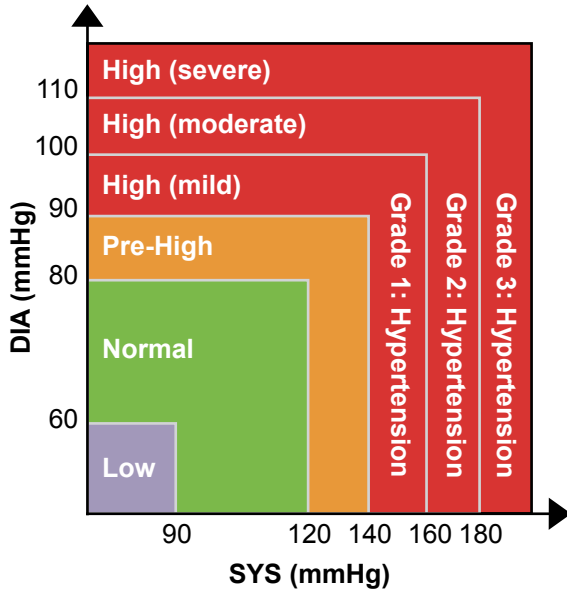
Table 2: Blood pressure ranges adults

	SYS	DIA
Low	≤89	≤59
Normal	90-120	60-80
Pre-High	121-140	81-90
High (mild)	141-160	91-100
High (moderate)	161-179	101-109
High (severe)	≥180	≥110

Caution:

- If your SYS reading exceeds 180 or DIA reading exceeds 110, immediately seek emergency medical help.

Troubleshooting

Error message	Cause
"E01"	Arm or body movement detected. The high pressure value can not be measured. Please redo the wrist cuff and test again.
"E02"	Arm or body movement detected. The low pressure value can not be measured. Please redo the wrist cuff and test again.
"E03"	Test results are not normal. Please measure again.
"E04"	The pulse signal is too weak. Please check the wrist cuff, then try again.


Problem	Possible Reason	Solution
No display when pressing the Start/Stop button	The batteries are low	Replace batteries
	The batteries are inserted incorrectly	Install in accordance with the positive and negative poles of the batteries
The display turns off during the reading	The batteries are low	Replace batteries
No display when pressing the Start/Stop button	The wrist cuff is not parallel to the heart	Measure with correct posture
	The wrist cuff is not applied properly	Reapply the cuff, then re-measure
	Speaking or moving during the reading	Be quiet and still, then re-measure
	Clothes are obstructing the reading	Remove the clothes, then re-measure
No pressure in the cuff	The wrist cuff is damaged or has a leak	Contact Duronic
Consecutive measurement results are very different	Mental state can cause fluctuations in blood pressure	Take a deep breath and keep in a quiet state, then re-measure
	Body/wrist movement	Remain still when taking the reading
	Not enough time between each reading	Wait at least 3 minutes between readings

For any other issues, please contact Duronic.

Technical Specifications

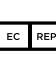Display Information	Display Mode (LCD)
Systolic blood pressure (SYS)	Kpa/mmHg
Systolic blood pressure (DIA)	Kpa/mmHg
Pulse Rate (Pulse/min)	bpm
Average (AVG)	3 saved readings
Memory	1 user (99 set storage)
Arrhythmia	Irregular heartbeat detected
NIBP (Non Invasive Blood Pressure) Parameter Specification	
Measuring range	0-280mmHg
Accuracy	≤±3mmHg(±0.4kPa)
PR Pulse Parameter Specification	
Measuring range	40-199 bpm
Accuracy	±5%
Wrist Cuff Specification	
13.5cm-19.5cm	
Battery Requirement	
2 X 1.5V (AAA size) alkaline battery	
Power off	
The appliance will automatically turn off after 30 seconds.	
Dimensions and Weight	
Dimensions	79 x 62 x 63 mm
Weight	About 78g (without batteries)

Key of Symbols

	Type BF applied part		Keep away from sunlight
	Date of manufacture		Serial number
	Manufacturer		Separate collection for electrical and electronic equipment
	Use-by date	IP22	Protected against solid foreign objects greater than 12.5mm in diameter and dripping water when tilted up to 15°
	Caution		
	Medical device		This device has no alarm function
	CE mark 0123		Not made with natural rubber latex
	Authorized representative in the EU		Keep dry
	Mobius Loop		Refer to instruction manual
	Restriction of Hazardous Substances		Green Dot

Device Specifications

Classification:

Class IIa (MDR) 2017/745

EMC complies with IEC60601-1 standard

Anti-electric Shock Type: Internally powered equipment

Anti-electric Shock Degree: Type BF equipment

CISPR 11 Group 1 Class B

Environment Requirements

Operation Temperature:	5°C ~ 40°C
Storage Temperature:	-20°C ~ 55°C
Ambient Humidity:	15%-80% RH, no condensation in operation 15%-95% RH, no condensation in storage
Atmospheric Pressure:	86kPa ~ 105 kPa, in operation 50kPa ~ 105 kPa, in storage

Xuzhou Yongkang Electronic Science Technology Co., Ltd

1st & 2nd Floor, 6#01, 6#02 | No.6 Building 1st Phase Economic Development
Manufacturing Zone | LIANDO U Valley | No.6 Leye Road | Xuzhou ETDZ |
221000 Xuzhou | People's Republic of China

**Prolinx GmbH**

Brehmstr. 56 | 40239 Düsseldorf | Germany

MPN: YK-BPW2**UK Distributor: Shine-Mart Ltd**

1 Spielsby Road | Romford | RM3 8SB | UK

**EU Distributor: Shinemart GmbH**

Brandenburger Str. 40 | 40880 Ratingen | DE

Appendix

Table 1: Guidance and manufacture’s declaration – electromagnetic emissions-for all EQUIPMENT and SYSTEMS

Guidance and manufacture’s declaration – electromagnetic emission		
The Duronic BPM65 is intended for use in the electromagnetic environment specified below. The customer or the user of the BPM65 should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The BPM65 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The BPM65 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	N/A
Voltage fluctuations / flicker emissions IEC 61000-3-3	N/A	N/A

Appendix

Table 2: Guidance and manufacturer’s declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacturer’s declaration – electromagnetic immunity		
The Duronic BPM65 is intended for use in the electromagnetic environment specified below. The customer or the user of the BPM65 should assure that it is used in such an environment.		
Immunity test	EN 60601 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	± 8 kV contact ± 15kV air
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	N/A
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	N/A
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles < 5 % U_T (>95 % dip in U_T) for 5 sec	N/A
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m

Table 3: Guidance and manufacturer’s declaration – electromagnetic immunity – for EQUIPMENT and SYSTEM that are not LIFE-SUPPORTING

Guidance and manufacturer’s declaration – electromagnetic immunity		
The Duronic BPM65 is intended for use in the electromagnetic environment specified below. The customer or the user of the BPM65 should assure that it is used in such an environment.		
Immunity test	EN 60601 test level	Compliance level
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A

Appendix

Table 4: Test specifications for enclosure port immunity to RF wireless communications equipment

Test frequency (MHz)	Band ^a (MHz)	Service ^a	Modulation ^b	Max power (W)	Distance (m)	Immunity test level
385	380 -390	TETRA 400	Pulse modulation ^b	1,8	0.3	27
450	430 - 470	GMRS 460, FRS 460	FM ^c ± 5 kHz deviation 1 kHz sine	2	0.3	28
710	704 - 787	LTE Band 13, 17	Pulse modulation ^b 217 Hz	0,2	0.3	9
745						
780						
810	800 - 960	GSM 800/900. TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation ^b 18 Hz	2	0,3	28
870						
930						
1 720	1700 - 1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3 4, 25; UMTS	Pulse modulation ^b 217 Hz	2	0,3	28
1 845						
1 970						
2 450	2400 - 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation ^b 217 Hz	2	0,3	28
5 240	5100 - 5800	WLAN 802.11 a/n	Pulse modulation ^b 217 Hz	0,2	0.3	9
5 500						
5 785						

NOTE: If necessary to achieve the immunity test level, the distance between the transmitting antenna and the me equipment or me system may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

- a) For some services, only the uplink frequencies are included.
- b) The carrier shall be modulated using a 50 % duty cycle square wave signal.
- c) As an alternative to FM modulation, 50 % pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

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EU Distributor: Shinemart GmbH

Brandenburger Str. 40 | 40880 Ratingen | DE

UK Information for consumers on the disposal of Waste Electrical & Electronic Equipment (WEEE).

This mark on a product and/or accompanying documents indicates that when it is to be disposed of, it must be treated as Waste Electrical & Electronic Equipment, (WEEE).

Any WEEE marked products must not be mixed with general household waste, but kept separate for the treatment, recovery and recycling of the materials used. For proper treatment, recovery and recycling; please take all WEEE marked waste to your Local Authority Civic waste site, where it will be accepted free of charge.

If customers dispose of Waste Electrical & Electronic Equipment correctly, they will be helping to save valuable resources and preventing any potential negative effects upon human health and the environment, of any hazardous materials that the waste may contain.

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