

INSTRUCTIONS FOR USE

EN

STEP 1: INSERTING THE BATTERY

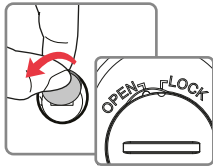
Using a coin, turn the battery cover anticlockwise from 'LOCK' to 'OPEN' and remove the battery cover.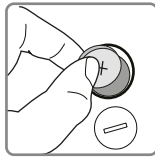


Insert battery with + mark facing up. Lock battery cover into position by turning the cover clockwise.

STEP 2: GEL PAD APPLICATION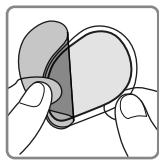



Remove the blue protective film from one side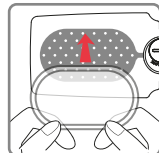

Place the Gel Pad (sticky side down) on to the grey electrode-area, on the device.

Repeat with the second Gel Pad.

STEP 3: USING REVITIVE PAIN RELIEVER

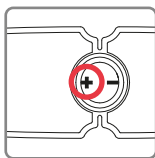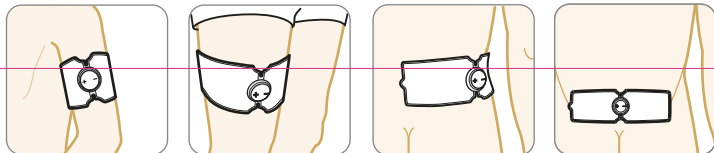Clean and dry the area you want to apply the stimulation to. Place REVITIVE onto the chosen area, pressing the adhesive pads against the skin. Press and hold + button for 3 seconds to power the unit on and listen for the beep. Press + button again to activate stimulation. Use +/- button to adjust stimulation level. REVITIVE switches itself off automatically after 20 minutes. If REVITIVE is not functioning properly or causing discomfort, stop using it immediately. Press and hold the - button for 3 seconds to turn the device OFF before removing the device from the skin

NB Device only works when in contact with the skin. When the device is not in contact with the skin, the device will not send out stimulation.

STEP 4: STORAGE

- Always place the blue plastic protective liners back onto the Gel Pads
- Store REVITIVE in a dry, dust-free and cool location, away from direct sunlight
- If REVITIVE will not be in use for more than 10 days, remove the battery

STEP 5: MAINTENANCE

Replacing the battery - REVITIVE Pain Reliever requires one CR2032 Lithium battery to operate. **Gel Pads** - Should be replaced regularly. Dry or dirty pads can cause skin irritation. Replacement Gel Pads can be obtained via the REVITIVE website.

REVITIVE PAIN RELIEVER

EN

READ ALL INSTRUCTIONS BEFORE USE

USE ONLY AS DIRECTED If symptoms persist, consult a healthcare professional.

Consult your doctor before using this device if you are unsure about the cause of your symptoms.

DO NOT USE IF (contraindications):

- You are fitted with an electronic implanted device, such as a heart pacemaker, Automatic Implantable Cardioverter Defibrillator (AICD) or insulin pump, or over a metallic implant
- You have a history of heart problems
- You are pregnant
- You have suspected or diagnosed epilepsy

WARNINGS:

- For external use only
- Pain relief may vary between users
- Do NOT use on your head, face, neck or chest
- Keep out of the reach of children
- Do NOT apply device directly on
 - open wounds or rashes, swollen, red, infected, or inflamed areas or skin eruptions (such as phlebitis, thrombophlebitis, varicose veins, cellulitis) or areas that lack sensation
 - (or close to) malignant tumours
 - areas treated with radiotherapy (within the past 6 months)
 - reproductive organs
- Do not use unit while sleeping, driving and do not allow unit to come into contact with water or use in places with high humidity eg bathroom
- There is a chance you may experience skin irritation or burns beneath the stimulation electrodes or hypersensitivity due to the electrical stimulation. Discontinue use and consult a healthcare professional if problems persist
- If your pain does not improve, gets worse or continues for more than 5 days, stop using the device and seek advice from your doctor
- Do not let the device touch any metal object, such as jewellery or belt buckles, during use
- Do not disassemble or attempt to repair
- Do not use Gel Pads after their use by date
- Single person use only – do not share with others
- If you experience adverse reactions, stop using REVITIVE and consult with your doctor
- Operation in close proximity (eg 1 m) to shortwave or microwave therapy medical equipment may produce instability in the REVITIVE Pain Reliever output.

INDICATIONS FOR USE

- Intended for temporary pain relief.
- Place REVITIVE Pain Reliever on the area of pain that you want to relieve (with the exception of areas listed in the warnings).
- Designed for the lower back, shoulders, arms and legs.

CONTENTS:

- REVITIVE Pain Reliever
- Self-Adhesive Gel Pads
- CR2032 Lithium Battery

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Copyright © 2018 Actegy Ltd. All rights reserved. ACTEGY & REVITIVE, are trademarks or registered trademarks of Actegy Ltd. The ACTEGY product is a proprietary design and is protected by applicable design laws.
Hivox Biotek B.V. De Run 4428, 5503 LR Velldhoven, Netherlands. Model: Painpad SP610
UK: Actegy Ltd. Reflex, Cain Road, Bracknell, RG12 1HL, United Kingdom.
FR/NL/BE: REVITIVE SAS, 75007 Paris. DE: Actegy GmbH, 10789 Berlin
AU: Actegy Pty Ltd, Unit 4 / 1 Newspaper Place, Maroochydore, QLD, 4558.
CA: Actegy Health Inc. Rosemont, IL 60018, USA.